

NobelSpeedy® Groovy

Instructions for use



Important: Please read.

Disclaimer of liability:

This product is part of an overall concept and may only be used in conjunction with the associated original products according to the instructions and recommendation of Nobel Biocare. Non-recommended use of products made by third parties in conjunction with Nobel Biocare products will void any warranty or other obligation, express or implied, of Nobel Biocare. The user of Nobel Biocare products has the duty to determine whether or not any product is suitable for the particular patient and circumstances. Nobel Biocare disclaims any liability, express or implied, and shall have no responsibility for any direct, indirect, punitive or other damages, arising out of or in connection with any errors in professional judgment or practice in the use of Nobel Biocare products. The user is also obliged to study the latest developments in regard to this Nobel Biocare product and its applications regularly. In cases of doubt, the user has to contact Nobel Biocare. Since the utilization of this product is under the control of the user, they are his/her responsibility. Nobel Biocare does not assume any liability whatsoever for damage arising thereof. Please note that some products detailed in this Instruction for Use may not be regulatory cleared, released or licensed for sale in all markets.

Description:

Implant:

NobelSpeedy® Groovy dental implants with external hexagon connection are made from biocompatible commercially pure grade 4 titanium with TiUnite® surface. The NobelSpeedy® Groovy is a slightly tapered implant, which gives a higher initial stability compared with a parallel implant.

Cover screw is not included.

Tooling:

Nobel Biocare Twist Drills, Twist Step Drills, Counterbores and Screw Taps are made of stainless steel with DLC (Diamond Like Carbon) coating and should be used in conjunction with NobelSpeedy® Groovy implants.

Intended use:

NobelSpeedy® Groovy endosseous implants are intended to be used in the upper or lower jaw bone (osseointegration) and used for anchoring or supporting tooth replacements to restore chewing function.

Indications for use:

NobelSpeedy® Groovy implant restorations range from single tooth to fixed-removable full dental arch overdenture applications to restore chewing function. This can be achieved by a 2-stage or 1-stage surgical technique in combination with immediate, early or delayed loading protocols, recognizing sufficient primary stability and appropriate occlusal loading for the selected technique. Implants allow also for bi-cortical anchorage in cases of reduced bone density to obtain high initial stability.

NobelSpeedy® Groovy implants with lengths of 20mm, 22mm and 25mm are indicated to be used in the upper jaw and soft bone only.

Contraindications:

It is contraindicated placing NobelSpeedy® Groovy implants in patients:

- who are medically unfit for an oral surgical procedure.
- with inadequate bone volume unless an augmentation procedure can be considered.
- whom adequate sizes, numbers or desirable position of implants are not reachable to achieve safe support of functional or eventually parafunctional loads.
- allergic or hypersensitive to commercially pure titanium (grade 4), stainless steel or DLC (Diamond Like Carbon) coating.

Warnings:

Failure to recognize actual lengths of drills relative to radiographic measurements can result in permanent injury to nerves or other vital structures. Drilling beyond the depth intended from lower jaw surgery may potentially result in permanent numbness to the lower lip and chin or lead to a hemorrhage in the floor of the mouth.

Besides the mandatory precautions for any surgery such as asepsis, during drilling in the jaw bone, one must avoid damage the nerves and vessels by referring to anatomical knowledge and preoperative radiographs.

Cautions:

General:

One hundred percent implant success cannot be guaranteed. Especially, non-observance of the indicated limitations of use and working steps may result in failure.

Treatment by means of implants may lead to loss of bone, biologic or mechanical failures including fatigue fracture of implants.

Close cooperation between surgeon, restorative dentist and dental laboratory technician is essential for a successful implant treatment.

It is strongly recommended that NobelSpeedy® Groovy implants are used only with Nobel Biocare surgical instruments and prosthetic components, as combining components that are not dimensioned for correct mating can lead to mechanical and/or instrumental failure, damage to tissue or unsatisfactory esthetic results.

It is strongly recommended that clinicians, new as well as experienced implant users, always go through special training before undertaking a new treatment method.

Nobel Biocare offers a wide range of courses for various levels of knowledge and experience.

For more info please visit www.nobelbiocare.com.

Working the first time with a colleague, experienced with the new device/treatment method, avoids eventual complications. Nobel Biocare has a global network of mentors available for this purpose.

Before surgery:

Careful clinical and radiological examination of the patient has to be performed prior to surgery to determine the psychological and physical status of the patient.

Special attention has to be given to patients who have localized or systemic factors that could interfere with the healing process of either bone or soft tissue or the osseointegration process (e.g., cigarette smoking, poor oral hygiene, uncontrolled diabetes, oro-facial radiotherapy, steroid therapy, infections in the neighboring bone). Special caution is advised in patients who receive bisphosphonate therapy.

In general, implant placement and prosthetic design must accommodate individual patient conditions. In case of bruxism or unfavorable jaw relationships reappraisal of the treatment option may be considered.

With respect to pediatric patients, routine treatment is not recommended until the end of the jaw bone growth phase has been properly documented.

Pre-operative hard tissue or soft tissue deficits may yield a compromised esthetic result or unfavorable implant angulations.

At surgery:

Particular caution should be used when placing narrow platform implants in the posterior region due to risk of prosthetic overload.

All instruments used in surgery must be maintained in good condition and care must be taken that instrumentation does not damage implants or other components.

NobelSpeedy® Groovy implants may be tilted up to 45° relative to the occlusal plane. When used with angulations between 30° and 45°, the following applies: The tilted implant must be splinted; a minimum of 4 implants must be used when supporting a fixed prosthesis in a fully edentulous arch.

After the implant installation, the surgeon's evaluation of bone quality and primary stability will determine when implants may be loaded. Lack of adequate quantity and/or quality of remaining bone, infection and generalized diseases may be potential causes for failure of osseointegration both immediately after surgery, or after osseointegration is initially achieved.

After surgery:

To secure the long term treatment outcome it is advised to provide comprehensive regular patient follow up after implant treatment and to inform about appropriate oral hygiene.

Surgical procedure:

During drilling procedures bone quality should be considered, please see table A: recommended drill sequences are based on bone quality to ensure optimal primary stability when applying immediate function.

The recommended drill sequence is based on bone quality. Drill data are stated in mm and the drills within brackets denote widening of cortex only. Counterbores and Screw Taps are available if deemed necessary.

No Screw Taps are available for NobelSpeedy RP 20mm, 22mm and 25mm implants.


A

Platform	Implant diameter	Drill Sequence (according to bone quality)		
		Soft bone Type IV	Medium bone Type II-III	Dense bone Type I
NP	3.3	Ø2.0	Ø2.0	Ø2.0 Ø2.4/2.8
RP	4.0	Ø2.0 (Ø2.4/2.8)	Ø2.0 Ø2.4/2.8 Ø3.2	Ø2.0 Ø2.4/2.8 Ø3.4
RP	5.0	Ø2.0	Ø2.0	Ø2.0
WP		Ø2.4/2.8 Ø3.0	Ø2.4/2.8 Ø3.2/3.6	Ø2.4/2.8 Ø3.2/3.6 Ø3.8/4.2
WP	6.0	Ø2.0 Ø2.4/2.8 Ø3.2/3.6	Ø2.0 Ø2.4/2.8 Ø3.2/3.6 Ø3.8/4.2	Ø2.0 Ø2.4/2.8 Ø3.2/3.6 Ø3.8/4.2 Ø5.0

Drilling must proceed at high speed (max. 2'000rpm for Twist Drills and Twist Step Drills) under constant and profuse irrigation by sterile saline at room temperature. In dense bone situation drill with continuous back and forth motion.

Depth measurement system: The parallel drills have a true depth measurement system. All drills and components are marked to prepare the site to the correct depth and obtain a secure and predictable position.

Caution: Twist Drills and Twist Step Drills extend up to 1 mm longer than the implant when seated. Allow for this additional length when drilling near vital anatomical structures (please see image B for drill reference lines).

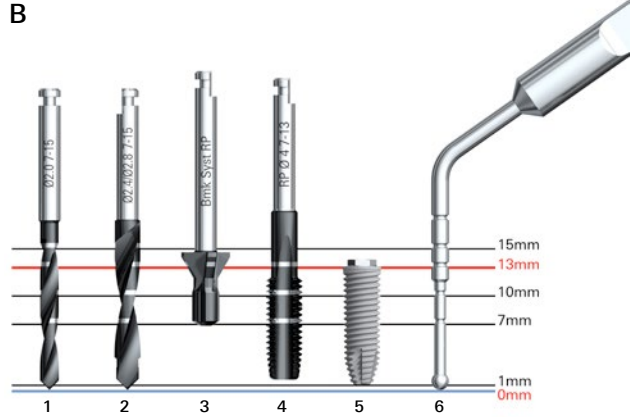

Drill reference lines for Twist Drills 7–15mm (1), Twist Step Drill 7–15mm (2), Counterbore (3), Screw Tap (4), NobelSpeedy® Groovy implant 13mm (5) and Depth Probe 7–18mm (6).

Twist Drills and Twist Step Drills are available in four different lengths with depth markings for 7–10mm, 7–15mm, 10–18mm and 18 – 25mm implants.

Screw Taps are available with depth markings for NP 7–15mm, RP and WP 7–13mm and 7–18mm implants.


Note: The marks on the Twist Drills and Twist Step Drills indicate actual millimeter lengths and correspond to the implant collar. Final vertical positioning depends on several clinical parameters, including esthetics, tissue thickness and available vertical space.

In situations where adjacent natural teeth interfere with the contra-angle head preventing the drill from reaching the desired depth, a drill extension shaft may be used.

1. Prepare implant site (C). When using a flapless approach add-on soft tissue height to drill depth.



2. Measure the final depth of implant site for applicable implant length using depth probe with same measurements as Twist Drills and Twist Step Drills.
3. Open the implant package and pick up the implant from inner casing with implant driver (please see D). The implants are ideally installed with low speed, max. 25rpm using a drill device (D) or Manual Torque Wrench Surgical (E).

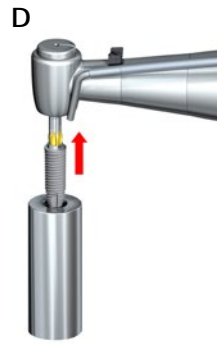

4. Place and tighten the implant using max. **45Ncm** insertion torque.

Caution: Never exceed insertion torque of **45 Ncm** for the implants. Overtightening an implant may lead to damage of the implant, fracture or necrosis of the bone site.

If the implant gets stuck during implant installation or **45Ncm** of insertion torque is achieved before fully seated, rotate the implant counter clockwise using drill device (reverse mode) or Manual Torque Wrench Surgical and remove implant from site. Replace the implant back into inner casing before proceeding further.

5. Dense bone protocol to be used when implant will not be fully seated.

- a. In cases of a thick cortical layer or dense bone a Counterbore and/or a Screw Tap is recommended to be able to get the implant fully seated and to release pressure around the implant neck.
- b. Select the Screw Tap matching the diameter of the implant.
- c. Place the Screw Tap into prepared implant site using low speed 25rpm and drill to appropriate depth. Switch the drill device with handpiece or Manual Torque Wrench Surgical to reverse mode and remove the Screw Tap.

Continue with implant installation until desired position is achieved using max. **45Ncm** installation torque.




6. For Immediate Function, the implant should be able to withstand a final torque of **35–45 Ncm**.
7. Depending on surgical protocol of choice, place a Cover Screw or Abutment and suture (F).



See table for implant specifications for NobelSpeedy® Groovy (G).

G

NobelSpeedy® Groovy

Platform	Platform diameter	Implant diameter	Lengths
	Ø 3.5 mm	Ø 3.3 mm	7 mm, 8.5 mm, 10 mm, 11.5 mm, 13 mm, 15 mm
	Ø 4.1 mm	Ø 4.0 mm Ø 5.0 mm	7 mm, 8.5 mm, 10 mm, 11.5 mm, 13 mm, 15 mm, 18 mm, 20 mm, 22 mm, 25 mm
	Ø 5.1 mm	Ø 5.0 mm Ø 6.0 mm	7 mm, 8.5 mm, 10 mm, 11.5 mm, 13 mm, 15 mm, 18 mm

For additional information on surgical procedures please consult the NobelSpeedy® Groovy “Procedures & products” treatment guidelines available at www.nobelbiocare.com or request latest printed version from a Nobel Biocare representative

Materials:

NobelSpeedy® Groovy implant: commercially pure titanium grade 4.

Twist Drills, Twist Step Drills, Counterbores and Screw Taps: stainless steel with DLC (Diamond Like Carbon) coating.

Cleaning and sterilization instructions:

NobelSpeedy® Groovy implants, Twist Drill, Twist Step Drills, Counterbores and Screw Taps are delivered sterile and for single use only prior to the labeled expiration date.

Warning: Do not use device if the packaging has been damaged or previously opened.

Caution: Implants, Twist Drills, Twist Step Drills, Counterbores and Screw Taps are single use products not intended to be reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.

MR safety information:

MR Conditional:

Non-clinical testing has demonstrated that the NobelSpeedy® Groovy implant is MR Conditional. A patient with this device can be safely scanned in an MR system meeting the following conditions:

- Static magnetic field of 1.5 Tesla and 3.0 Tesla only.
- Maximum spatial gradient magnetic field of 4000 Gauss/cm (40 T/m).
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 4 W/kg (First Level Controlled Mode).

Under the scan conditions defined above, the NobelSpeedy® Groovy implant is expected to produce a maximum temperature rise of 4.1°C after 15 minutes of continuous scanning.

In non-clinical testing, the image artifact caused by the device extends approximately 30mm from the NobelSpeedy® Groovy implant when imaged with a gradient echo pulse sequence and a 3.0 Tesla MRI system.

Removable restorations should be taken out prior to scanning, as is done for watches, jewelry etc.

Storage and handling:

The product must be stored in a dry place in the original packaging at room temperature and not exposed to direct sunlight. Incorrect storage may influence device characteristics leading to failure.

Disposal:

Disposal of the device shall follow local regulations and environmental requirements, taking different contamination levels into account.

Manufacturer: Nobel Biocare AB, Box 5190, 402 26
 Västra Hamngatan 1, 411 17 Göteborg, Sweden.
 Phone: +46 31 81 88 00. Fax: +46 31 16 31 52. www.nobelbiocare.com



Canada license exemption: Please note that not all products may have been licensed in accordance with Canadian law.

Prescription device: Rx only

Caution: Federal law restricts this device to sale by or on the order of a licensed physician or dentist.

Symbols Glossary:

The following symbols may be present on the device labeling or in information accompanying the device. Refer to the device labeling or accompanying information for the applicable symbols.



Authorized representative in the European Community



Batch code



Catalogue number



Caution



CE marking



Consult instructions for use



Contains hazardous substances



Contains or presence of phthalate



Date



Date of manufacture



Do not re-sterilize



Do not re-use



Do not use if package is damaged



Double sterile barrier system

Rx Only

For prescription use only



Health care centre or doctor



Keep away from sunlight



Keep dry

symbol.glossary.nobelbiocare.com
ifu.nobelbiocare.com

Link to Online Symbols Glossary and IFU Portal



Magnetic resonance conditional



Manufacturer



Medical device



Non-pyrogenic



Non-sterile



Patient identification



Patient information website



Patient number



Serial number



Single sterile barrier system



Single sterile barrier system with protective packaging inside



Single sterile barrier system with protective packaging outside



Sterilized using ethylene oxide



Sterilized using irradiation



Temperature limit



Tooth number



Upper limit of temperature



Sterilized using steam or dry heat



Unique Device Identifier



Use-by date

EN All rights reserved.

Nobel Biocare, the Nobel Biocare logotype and all other trademarks used in this document are, if nothing else is stated or is evident from the context in a certain case, trademarks of Nobel Biocare. Product images in this folder are not necessarily to scale. All product images are for illustration purposes only and may not be an exact representation of the product.