


Replace Select™ Tapered TiUnite®, Replace Select™ Tapered PMC

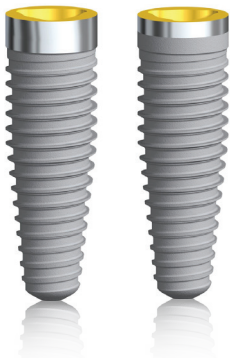

Important – Disclaimer of Liability

This product is part of an overall concept and may only be used in conjunction with the associated original products according to the instructions and recommendation of Nobel Biocare. Non-recommended use of products made by third parties in conjunction with Nobel Biocare products will void any warranty or other obligation, express or implied, of Nobel Biocare. The user of Nobel Biocare products has the duty to determine whether or not any product is suitable for the particular patient and circumstances. Nobel Biocare disclaims any liability, express or implied, and shall have no responsibility for any direct, indirect, punitive or other damages, arising out of or in connection with any errors in professional judgment or practice in the use of Nobel Biocare products. The user is also obliged to study the latest developments in regard to this Nobel Biocare product and its applications regularly. In cases of doubt, the user has to contact Nobel Biocare. Since the utilization of this product is under the control of the user, they are his/her responsibility. Nobel Biocare does not assume any liability whatsoever for damage arising thereof.

Please note that some products detailed in this Instruction for Use may not be regulatory cleared, released or licensed for sale in all markets.

Description

Replace Select™ Tapered TiU (TiUnite®) and Replace Select™ Tapered PMC (Partially Machined Collar) dental implants are made from commercially pure titanium (grade 4).

- Implant macroshape: tapered body
- Implant connection: tri-channel
- Implant surface:
 - Replace Select™ Tapered TiU: TiUnite® anodized implant surface and 1.5 mm machined collar
 - Replace Select™ Tapered PMC: TiUnite® anodized implant surface and 0.75 mm machined collar
- Color-coding: the implant platform is anodized with the color-coding of the prosthetic platform

Replace Select™ Tapered TiU and Replace Select™ Tapered PMC include a co-packed Cover Screw made of titanium alloy Ti-6Al-4V. For information specific to the Cover Screws, refer to the Nobel Biocare Instructions for Use for the component (IFU1016).

Table 1 describes the compatibility of the products with each other.

Table 1 – Compatibility with Accessories or Other Devices

| Product Name | Cover Screw | Implant Driver | Clinical Screws | Abutments | | | Impression Coping | Other |
|-----------------------------|---|---------------------------|--|-----------------------------|--|--|------------------------------------|---|
| | | | | Healing | Temporary | Permanent | | |
| Replace Select™ Tapered NP | Cover Screw NobRpl NP Healing Screw RplSel TC NP | Implant Driver NobRpl NP | Abutment Screw NobelReplace® NP Screw Multi-unit Angled Abutment NobRpl NP Screw Ceramic Abutment NobRpl NP | Healing Abutment NobRpl NP | Temporary Abutment Engaging NobRpl NP Temporary Abutment Non-Eng NobRpl NP Immediate Temporary Abutment NobRpl NP Plastic/Temp Coping Esthetic Abutment NP Plastic/Temp Coping 15° Esthetic Ab NP | Snappy Abutment NobRpl NP Esthetic Abutment NobRpl NP 15° Esthetic Abutment NobRpl NP Universal Base Tri-Channel NP Universal Base Burn-out Coping NP/RP GoldAdapt™ Engaging NobRpl NP GoldAdapt™ Non-Engaging NobRpl NP Multi-unit Abutment NobRpl NP 17° Multi-unit Abutment NobRpl NP Locator Abut for 3.5 RPL Sel Impl. Ball Abutment Titanium NobRpl NP NobelProcera® Ab Ti NobelReplace® NP Procera® Abutment Zr NobRpl NP | Impr Coping Open Tray NobRpl NP | PIB IBO Bone Mill with Guide NobRpl NP Bone Mill Guide NobelReplace® NP Impl Retr Instr Hex & Tri-Ch NP/RP Implant Rescue Collar Tri-Channel Ø3.5 |
| Replace Select™ Tapered RP | Cover Screw NobRpl RP Healing Screw RplSel TC RP | Implant Driver NobRpl RP | Abutment Screw NobRpl RP/WP/6.0 Screw Multi-unit Angled Abutment NobRpl RP Screw Ceramic Abutment NobRpl RP/WP/6.0 | Healing Abutment NobRpl RP | Temporary Abutment Engaging NobRpl RP Temporary Abutment Non-Eng NobRpl RP Immediate Temporary Abutment NobRpl RP | Snappy Abutment NobRpl RP Esthetic Abutment NobRpl RP 15° Esthetic Abutment NobRpl RP Universal Base Tri-Channel RP Universal Base Burn-out Coping NP/RP GoldAdapt™ Engaging NobRpl RP GoldAdapt™ Non-Engaging NobRpl RP Multi-unit Abutment NobRpl RP 17° Multi-unit Abutment NobRpl RP 30° Multi-unit Abutment NobRpl RP Locator Abut for 4.3 RPL Sel Impl. Ball Abutment Titanium NobRpl RP 3mm NobelProcera® Ab Ti NobelReplace RP Procera® Abutment Zr NobRpl RP | Impl Retr Instr Hex & Tri-Ch NP/RP | PIB IBO Bone Mill Guide NobelReplace® RP Bone Mill with Guide NobRpl RP Impl Retr Instr Hex & Tri-Ch NP/RP Implant Rescue Collar Tri-Channel Ø4.3 |
| Replace Select™ Tapered WP | Cover Screw NobRpl WP | Implant Driver NobRpl WP | Screw Ceramic Abutment NobRpl RP/WP/6.0 | Healing Abutment NobRpl WP | Temporary Abutment Engaging NobRpl WP Temporary Abutment Non-Eng NobRpl WP Temp Abutment Plastic Non-Eng NobRpl WP Immediate Temporary Abutment NobRpl WP Plastic/Temp Cop Esthetic Ab NobRpl WP Pl/Temp Coping 15° Esthetic Ab NobRpl WP | Snappy Abutment NobRpl WP Esthetic Abutment NobRpl WP 15° Esthetic Abutment NobRpl WP Universal Base Burn-out Coping WP Universal Base Tri-Channel WP GoldAdapt™ Non-Engaging NobRpl WP GoldAdapt™ Engaging NobRpl WP Multi-unit Abutment NobRpl WP Locator Abut for RPL Sel Impl. NobelProcera® Ab Ti NobelReplace WP Procera® Abutment Zr NobRpl WP | Impr Coping Open Tray NobRpl WP | PIB IBO Bone Mill Guide NobelReplace WP Bone Mill with Guide NobRpl WP Ø6.5 Impl Retr Instr CC RP & Tri-Ch WP Impl Retr Instr CC RP & Tri-Ch WP & TCC RP |
| Replace Select™ Tapered 6.0 | Cover Screw NobRpl 6.0 mm | Implant Driver NobRpl 6.0 | Screw Ceramic Abutment NobRpl RP/WP/6.0 | Healing Abutment NobRpl 6.0 | Temporary Abutment Engaging NobRpl 6.0 Temp Abutment Plastic Non-Eng NobRpl 6.0 Temporary Abutment Engaging NobRpl 6.0 Temporary Abutment Non-Eng NobRpl 6.0 Immediate Temporary Abutment NobRpl 6.0 | Snappy Abutment NobRpl 6.0 Esthetic Abutment NobRpl 6.0 15° Esthetic Abutment NobRpl 6.0 GoldAdapt™ Non-Engaging NobRpl 6.0 GoldAdapt™ Engaging NobRpl 6.0 NobelProcera® Ab Ti NobelReplace 6.0 Procera® Abutment Zr NobRpl 6.0 | Impr Coping Open Tray NobRpl 6.0 | PIB IBO Bone Mill Guide NobRpl 6.0 Bone Mill with Guide NobRpl 6.0 Ø7 Impl Retr Instr Tri-Channel 6.0 22mm |

Intended Use / Intended Purpose

Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants:

Intended for use as an endosseous dental implant in the maxilla or mandible for anchoring or supporting dental prostheses to restore chewing function.

Indications

Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implant restorations range from single tooth to fixed-removable full dental arch overdenture applications to restore chewing function. This can be achieved by a 2-stage or 1-stage surgical technique in combination with immediate, early or delayed loading protocols, recognizing sufficient primary stability and appropriate occlusal loading for the selected technique.

Contraindications

It is contraindicated to use Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants in:

- Patients who are medically unfit for an oral surgical procedure
- Patients with inadequate bone volume unless an augmentation procedure can be considered.
- Patients in whom adequate sizes, numbers or desirable positions of implants are not reachable to achieve safe support of functional or eventually parafunctional loads.
- Patients who are allergic or hypersensitive to commercially pure titanium (grade 4) or titanium alloy Ti-6Al-4V (90% titanium, 6% aluminum, 4% vanadium).

For contraindications specific to the Cover Screws, refer to the Nobel Biocare Instructions for Use for the component (IFU1016).

For contraindications specific to the Screw Drivers, refer to the Nobel Biocare Instructions for Use for the component (IFU1085).

For contraindications specific to the Manual Torque Wrenches Surgical and Prosthetic, refer to the Nobel Biocare Instructions for Use for the component (IFU1098).

For contraindications specific to the Implant Drivers, refer to the Nobel Biocare Instructions for Use for the component (IFU1090).

Materials

- Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants: Unalloyed Titanium grade 4 according to ASTM F67 and ISO 5832-2.
- Cover screw: Titanium alloy Ti-6Al-4V (90% titanium, 6% aluminum, 4% vanadium) according to ASTM F136 and ISO 5832-3.

Warnings

Failure to recognize actual lengths of drills relative to radiographic measurements can result in permanent injury to nerves or other vital structures.

Besides the mandatory precautions for any surgery such as of asepsis, during drilling in the jaw bone, one must avoid damage to nerves and vessels by referring to anatomical knowledge and preoperative radiographs.

Cautions

General

One hundred percent implant success cannot be guaranteed. In particular, non-observance of the product's indications for use and the surgical/handling procedure(s) may result in failure.

Treatment by means of implants may lead to loss of bone, biologic or mechanical failures including fatigue fracture of implants.

Close cooperation between surgeon, restorative dentist and dental laboratory technician is essential for a successful implant treatment.

Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants must only be used with compatible Nobel Biocare instruments and components and prosthetic components. Use of instruments or components or prosthetic components that are not intended to be used in combination with Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants can lead to product failure, damage to tissue, or unsatisfactory esthetic results.

When using a new device/treatment method for the first time, working with a colleague who is experienced with the new device/treatment method may help avoid possible complications. Nobel Biocare has a global network of mentors available for this purpose.

Before Surgery

Careful psychological and physiological evaluation, followed by clinical and radiological examination must be performed on the patient prior to surgery to determine the suitability of the patient for treatment.

Special attention has to be given to patients who have local or systemic factors that could interfere with the healing process of either bone or soft tissue or the osseointegration process (e.g., cigarette smoking, poor oral hygiene, uncontrolled diabetes, oro-facial radiotherapy, steroid therapy, infections in the neighboring bone). Special caution is advised in patients who receive bisphosphonate therapy.

In general, implant placement and prosthetic design must accommodate individual patient conditions. In case of bruxism, other parafunctional habits or unfavorable jaw relationships reappraisal of the treatment option may be considered.

The device has not been evaluated in pediatric/adolescent patients and is not recommended for use in children. Routine treatment is not recommended until the end of the juvenile jaw bone growth phase has been properly documented.

Pre-operative hard tissue or soft tissue deficits may yield a compromised esthetic result or unfavorable implant angulations.

All components, instruments and tooling used during the clinical and/or laboratory procedure must be maintained in good condition and care must be taken that instrumentation does not damage implants or other components.

At Surgery

Particular caution should be used when placing narrow platform implants in the posterior region due to risk of prosthetic overload.

Care and maintenance of sterile instruments are crucial for a successful treatment. Sterilized instruments not only safeguard your patients and staff against infection but are also essential for the outcome of the total treatment.

Because of the small sizes of the devices, care must be taken that they are not swallowed or aspirated by the patient. It is appropriate to use specific supporting tools to prevent aspiration of loose parts (e.g., dental dam, gauze, or throat shield).

The implants may be tilted up to 45° relative to the occlusal plane. When used with angulations between 30° and 45°, the following applies: The tilted implant must be splinted; a minimum of 4 implants must be used when supporting a fixed prosthesis in a fully edentulous arch.

After the implant installation, the surgeon's evaluation of bone quality and primary stability will determine when implants may be loaded. Lack of adequate quantity and/or quality of remaining bone, infection and generalized diseases may be potential causes for failure of osseointegration both immediately after surgery, or after osseointegration is initially achieved.

After Surgery

To help ensure a successful long term-treatment outcome, it is advised to provide comprehensive regular patient follow up after implant treatment and to inform about appropriate oral hygiene.

Intended Users and Patient Groups

Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants are to be used by dental health care professionals.

Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants are to be used in patients subject to dental implant treatment.

Clinical Benefits and Undesirable Side Effects

Clinical Benefits Associated with Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants

Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants are a component of treatment with a dental implant system and/or dental crowns and bridges. As a clinical benefit of treatment, patients can expect to have their missing teeth replaced and/or crowns restored.

Undesirable Side Effects Associated with Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants

The placement of a dental implant constitutes an invasive treatment which may be associated with typical side effects such as inflammation, infection, bleeding, hematoma, pain, and swelling. Depending on the location, placement of the implant may also lead (in rare cases) to bone fracture, damage/perforation of neighboring structures/restorations, sinusitis, or sensory/motor disturbances. During placement of this device the pharyngeal (gag) reflex may be triggered in patients with a sensitive gag reflex.

Dental implants are the substructure of a multi-component system that replaces teeth and as a result, the implant recipient may experience side effects similar to those associated with teeth, such as mucositis, calculus, peri-implantitis, fistulas, ulcers, soft tissue hyperplasia, soft and/or hard tissue recession/loss. Some patients may experience discoloration in the mucosal area such as graying.

Where required per the European Medical Device Regulation (MDR; EU 2017/745) a Summary of Safety and Clinical Performance document (SSCP) is available for the Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants. The SSCP can be obtained at the following website:

ec.europa.eu/tools/eudamed¹

¹ Website available upon launch of the European Database on Medical Devices (EUDAMED)

Notice regarding serious incidents

For a patient/user/third party in the European Union and in countries with an identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. The contact information for the manufacturer of this device to report a serious incident is as follows:

Nobel Biocare AB
www.nobelbiocare.com/complaint-form

Surgical Procedure

1. Drilling must proceed at high speed (maximum 800 rpm for Tapered Drills) under constant and profuse irrigation by sterile saline at room temperature. Tapered Drills are internally-irrigated and require a specific technique to prevent irrigation holes becoming plugged with bone debris. During drilling use an in-and-out motion and drill in bone for 1–2 seconds. Move the drill up without stopping the handpiece motor which allows the irrigation to flush away bone debris.

Caution Tapered Drills extend up to 1 mm longer than the implant when seated. Allow for this additional length when drilling near vital anatomical structures.

Figure A shows protocol steps and “Product Reference line” for tapered implants, 13 mm long with regular platform.

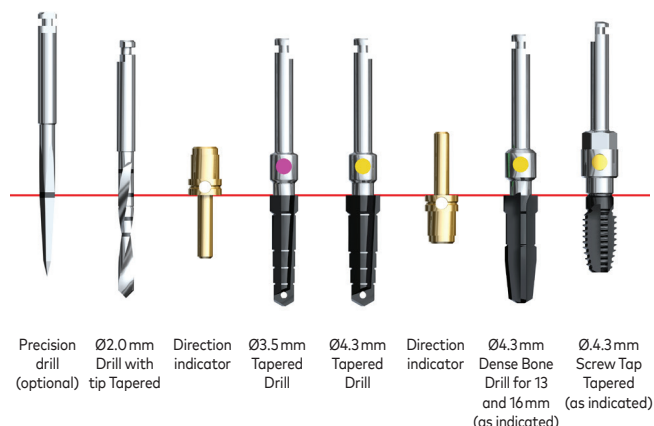


Figure A – Protocol steps

When using a flapless approach add on soft tissue height to drill depth.

In situations where adjacent structures (natural teeth) would interfere with the angle head and prevent the drill from reaching the desired depth, a drill extension shaft may be used.

2. Prepare implant site using Drill with Tip Tapered 2 mm (Figure B) and respective Tapered Drills depending on the implant to be installed, length and platform (Figure C).

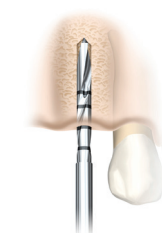


Figure B – Drill with Tip Tapered 2 mm



Figure C – Tapered Drills

3. Open the implant package and pick up the implant from the inner casing with the implant driver (Figure D). The implants are ideally installed with low speed (maximum 25 rpm) using a drilling device or the Manual Torque Wrench Surgical.

Place and tighten the implant using max 45 Ncm installation torque (Figure E).



Figure D – Implant driver picking up implant



Figure E – max 45 Ncm

To ensure ideal prosthetic abutment orientation, position one of the tri-channel lobes towards buccal/facial. To facilitate proper orientation, refer to the markings on the implant driver (Figure E).

Caution Never exceed insertion torque of 45 Ncm. Overtightening an implant may lead to damage of the implant, fracture or necrosis of the bone site. If a Surgical Driver is used to insert the implant, special care needs to be taken to avoid over tightening.

If the implant gets stuck during implant installation or 45 Ncm of insertion torque is achieved before fully seated, rotate the implant counter clockwise using the drilling device (reverse mode) or Manual Torque Wrench and remove the implant from site. Replace the implant back into the inner casing before proceeding further.

4. Dense bone protocol – as indicated:
 - a. Dense Bone Drill Tapered (Figure F) is only needed for 13 mm and 16 mm implants. If shorter implants are used, go directly to step c. Select the Dense Bone Drill matching the diameter and length (13 or 16 mm) of the final Tapered Drill.
 - b. Drill one pass into the prepared site with high speed (800 rpm) using the Dense Bone Drill.
 - c. For product reference line Screw Tap vs implant length see (Figure G). Select the Screw Tap Tapered matching the diameter of the final Tapered Drill. Place into prepared implant site using low speed (25 rpm).
 - d. Apply firm pressure and begin rotating the Screw Tap slowly. When the threads engage, allow the Screw Tap to feed without pressure to appropriate depth (Figure H).
 - e. Switch the handpiece to reverse mode and back the Screw Tap out.



Figure F – Dense Bone Drill



Figure G – Reference lines for implant length



Figure H – Screw Tap inserted

Continue with the implant installation until the desired position is achieved using max 45 Ncm installation torque.

5. For Immediate Function, the implant should be able to withstand a final torque of 35–45 Ncm.
6. Depending on surgical protocol of choice, place a cover screw or abutment and suture (Figure I, Figure J).



Figure I – Cover screw

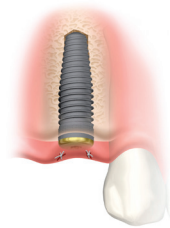






Figure J – Healing abutment

See Table 2 for implant specifications.

Table 2 – Implant specifications

| Platform | Implant diameter | Lengths |
|---|------------------|-------------------------------------|
|  | Ø 3.5 mm | 8 mm, 10 mm, 11.5 mm, 13 mm, 16 mm |
|  | Ø 4.3 mm | 8 mm, 10 mm, 11.5 mm, 13 mm, 16 mm |
|  | Ø 5.0 mm | 8 mm, 10 mm, 11.5 mm, 13 mm, 16 mm |
|  | Ø 6.0 mm | 8 mm, 10 mm, 11.5 mm, 13 mm, 16 mm* |

*16 mm implant is not available for Replace Select™ Tapered TiUnite®

For information specific to the Cover Screws, refer to the Nobel Biocare Instructions for Use for the component (IFU1016).

Sterility and Reusability Information

Caution Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants are delivered sterile for single use only. Do not use after the labeled expiration date.

Do not use device if the packaging has been damaged or previously opened as the device sterility and/or integrity may be compromised.

Warning Use of non-sterile device may lead to infection of tissues or infectious diseases.

Caution Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants are a single use product and must not be reprocessed. Reprocessing could cause loss of mechanical, chemical and / or biological characteristics. Reuse could cause local or systemic infection.

Magnetic Resonance (MR) Safety Information

MRI Safety Information



Non-clinical testing has demonstrated the Replace Select™ Tapered TiU and Replace Select™ Tapered PMC implants are MR conditional. A patient with this device can be safely scanned in an MR system meeting the following conditions mentioned here below. Failure to follow these conditions may result in injury to the patient.

| | | |
|---|---|--|
| Nominal value(s) of Static Magnetic Field [T] | 1.5-Tesla (1.5 T) | 3-Tesla (3 T) |
| Maximum Spatial Field Gradient [T/m and gauss/cm] | Maximum spatial field gradient of 44.4 T/m (4,440 G/cm). | |
| RF Excitation | Circularly Polarized (CP) | |
| RF Transmit Coil Type | Whole body transmit coil | |
| Maximum Whole-Body SAR [W/kg] | Inferior to the shoulders: 2.0 W/kg Superior to the shoulders: 0.2 W/kg | Inferior to the navel: 2.0 W/kg Superior to the navel: 0.1 W/kg |
| Limits on Scan Duration | Under the scan conditions defined above, the dental implant systems are expected to produce a maximum temperature rise less than 6.0 °C after 15 minutes of continuous scanning. | |
| MR Image Artifact | In non-clinical testing, the image artifact caused by the dental implant systems extend radially approximately 3 cm from the devices or device assemblies when imaged in a 3T MRI system. | |

Caution

Configurations with more than 2 Zygoma implants have not been evaluated for safety and compatibility in the MR environment. They have not been tested for heating, migration, or image artifact in the MR environment. The safety of configurations with more than 2 Zygoma implants in the MR environment is unknown. Scanning a patient who has this configuration may result in patient injury.

Performance Requirements and Limitations

To achieve the desired performance, Replace Select™ Tapered implants must only be used with the products described in this Instructions for Use and/or in the Instructions for Use for other compatible Nobel Biocare products, and in accordance with the Intended Use for each product. To confirm the compatibility of products which are intended to be used in conjunction with Replace Select™ Tapered implants, check the color coding, dimensions, lengths, connection type and/or any direct marking as applicable on the products or product labeling.

Facilities and Training

It is strongly recommended that new and experienced users of Nobel Biocare products always go through special training before using a new product for the first time. Nobel Biocare offers a wide range of courses for various levels of knowledge and experience. For more information please visit www.nobelbiocare.com.

Storage, Handling and Transportation




The device must be stored and transported in dry conditions in the original packaging at room temperature and not exposed to direct sunlight. Incorrect storage and transportation may influence device characteristics leading to failure.

Disposal

Safely discard potentially contaminated or no longer usable medical devices as healthcare (clinical) waste in accordance with local healthcare guidelines, country and government legislation or policy.

Separation, re-cycling or disposal of packaging material shall follow local country and government legislation on packaging and packaging waste, where applicable.

Manufacturer and Distributor Information

| | |
|--|--|
| Manufacturer  | Nobel Biocare AB PO Box 5190, 402 26 Västra Hamngatan 1 Göteborg 411 17 Sweden www.nobelbiocare.com |
| UK Responsible Person <div>UK RP</div> | Nobel Biocare UK Ltd 4 Longwalk Road Stockley Park Uxbridge UB11 1FE United Kingdom |
| Distributed in Turkey by | EOT Dental Sağlık Ürünleri ve Dış Ticaret A.Ş. Nispetiye Mah. Aytar Cad. Metro İş Merkezi No: 10/7 Beşiktaş İSTANBUL Phone: +90 2123614901, Fax: +90 2123614904 |
| Distributed in Australia by | Nobel Biocare Australia Pty Ltd Level 4, 7 Eden Park Drive Macquarie Park, NSW 2113 Australia Phone: +61 1800 804 597 |
| Distributed in New Zealand by | Nobel Biocare New Zealand Ltd 33 Spartan Road Takanini, Auckland, 2105 New Zealand Phone: +64 0800 441 657 |
| CE Mark for Class IIb Devices |  2797 |
| UKCA Mark for Class IIb Devices |  0086 |

Note Refer to the product label to determine the applicable conformity marking for each device.

Basic UDI-DI Information

| Product | Basic UDI-DI Number |
|-----------------------------|---------------------|
| Replace Select™ Tapered TiU | 733274700000012875 |
| Replace Select™ Tapered PMC | |

Implant Card

Replace Select™ Tapered implants are accompanied by an Implant Card which contains important information for patients regarding the device.

Complete the Implant Card by filling it out with the patient- and device-specific information as indicated and provide the completed Implant Card to the patient.

Legal Statements

EN All rights reserved.

Nobel Biocare, the Nobel Biocare logotype and all other trademarks used in this document are, if nothing else is stated or is evident from the context in a certain case, trademarks of Nobel Biocare. Product images in this folder are not necessarily to scale. All product images are for illustration purposes only and may not be an exact representation of the product.

Symbols Glossary

The following symbols may be present on the device labeling or in information accompanying the device. Refer to the device labeling or accompanying information for the applicable symbols.



Authorized Representative in the European Community / European Union



UK Responsible Person



Authorised Representative in Switzerland



Sterilized using Ethylene Oxide



Sterilized using irradiation



Sterilized using steam or dry heat



Batch code



Catalogue number



Unique Device Identifier



Serial number



Medical device



Magnetic resonance safe



Caution



Magnetic resonance conditional



Non-sterile



Contains hazardous substances



Contains or presence of DEHP phthalate



Contains or presence of natural rubber latex



Contains or presence of phthalate



Contains biological material of animal origin



CE mark



CE mark with Notified Body number



UKCA mark



UKCA mark with Approved Body number



Consult instructions for use



For prescription use only



symbol.glossary.nobelbiocare.com
ifu.nobelbiocare.com

Link to Online Symbols Glossary and IFU Portal



Date of manufacture



Manufacturer



Use-by date



Upper limit of temperature



Temperature limit



Do not re-sterilize



Do not re-use



Non-pyrogenic



Date



Tooth number



Patient number



Patient identification



Health care centre or doctor



Patient information website



EU Importer



Swiss Importer



Double sterile barrier system



Single sterile barrier system



Single sterile barrier system with protective packaging inside



Single sterile barrier system with protective packaging outside



Do not use if package is damaged and consult instructions for use



Keep away from sunlight



Keep dry