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IFU1007. Brånemark System® Mk III TiUnite®, NobelSpeedy®, Twist Drill, Twist Step Drill, Counterbore, Screw Tap

Nome prodotto	Vite di copertura	Driver per impianto	Viti	Abutment			Transfer per impronta
				Guarigione	Provvisorio	Permanente	
Brånemark System® Mk III TiU NP	Cover Screw Brånemark System® NP	Implant Driver NP Brånemark System Implant Driver Wrench Adapter NP	Abutment Screw Multi-unit Angled Prosthetic Screw Multi-unit	Healing Abutment NP	Immediate Temporary Abutment NP	Procera Esthetic Abutment NP	Impression Coping Closed Tray NP
					Plastic Coping Immediate NP	Esthetic Abutment NP	Impression Coping Open Tray NP
					Temporary Abutment NP	15° Esthetic Abutment NP	Impression Coping Open Tray
					QuickTemp™ Abutment Conical NP	Snappy™ Abutment NP	Multi-unit NP
					Plastic Coping QuickTemp™ Abutment Conical NP	Universal Base NP	Impression Coping Closed Tray
					Temporary Abutment Engaging NP	GoldAdapt™ Engaging NP	Multi-unit NP
					Temporary Abutment Non-Engaging NP	GoldAdapt™ Non-Engaging NP	Impression Coping Bar Closed Tray
					Temporary Abutment Plastic Engaging NP	Multi-Unit Abutment NP	Multi-unit NP
					Temporary Abutment Plastic Non-Engaging NP	17° Multi-Unit Abutment NP	Temporary Coping Multi-unit Titanium NP
							Temporary Coping Multi-unit Plastic NP
Brånemark System® Mk III TiU RP	Cover Screw Brånemark System® RP	Implant Driver RP Brånemark System® Implant Driver Wrench Adapter RP		Healing Abutment RP	Immediate Temporary Abutment RP	Procera Esthetic Abutment RP	Impression Coping Closed Tray RP
					Plastic Coping Immediate Temporary Abutment RP	Esthetic Abutment RP	Impression Coping Open Tray RP
					QuickTemp™ Abutment Conical RP	15° Esthetic Abutment RP	Impression Coping Bar Brånemark System® RP
					Plastic Coping QuickTemp™ Abutment Conical RP	Snappy™ Abutment RP	Impression Coping Open Tray
					Temporary Abutment Engaging RP	Universal Base RP	Multi-unit RP
					Temporary Abutment Non-Engaging RP	Burn-out Coping RP	Impression Coping Closed Tray
					Temporary Abutment Plastic Engaging RP	GoldAdapt™ Engaging RP	Multi-unit RP
					Temporary Abutment Plastic Non-Engaging RP	GoldAdapt™ Non-Engaging RP	Impression Coping Bar Closed Tray
						Multi-Unit Abutment RP	Multi-unit RP
						17° Multi-Unit Abutment RP	Temporary Coping Multi-unit Titanium RP
						30° Multi-Unit Abutment RP	Temporary Coping Multi-unit Plastic RP
						30° Multi-Unit Abutment Non-Engaging RP	
						Locator Abutment RP	
						Ball Abutment Titanium RP	
Brånemark System® Mk III TiU WP	Cover Screw Brånemark System® WP	Implant Driver WP Brånemark System Implant Driver Wrench Adapter WP	Prosthetic Screw Multi-unit	Healing Abutment WP	Immediate Temporary Abutment WP	Procera Esthetic Abutment WP	Impression Coping Closed Tray WP
					Plastic Coping Immediate Temporary Abutment WP	Esthetic Abutment WP	Impression Coping Open Tray WP
					QuickTemp™ Abutment Conical WP	15° Esthetic Abutment WP	Impression Coping Open Tray
					Plastic Coping QuickTemp™ Abutment Conical WP	Snappy™ Abutment WP	Multi-unit WP
					Temporary Abutment Engaging WP	Abutment WP	Impression Coping Closed Tray
					Temporary Abutment Non-Engaging WP	Universal Base WP	Multi-unit WP
					Temporary Abutment Plastic Engaging WP	Universal Base Burn-out Coping WP	Impression Coping Bar Closed Tray
					Temporary Abutment Plastic Non-Engaging WP	GoldAdapt™ Engaging WP	Multi-unit WP
						GoldAdapt™ Non-Engaging WP	Temporary Coping Multi-unit Titanium WP
						Multi-Unit Abutment WP	Temporary Coping Multi-unit Plastic WP
						Locator Abutment WP	

Nome prodotto	Vite di copertura	Driver per impianto	Viti	Abutment			Transfer per impronta
				Guarigione	Provvisorio	Permanente	
NobelSpeedy® Shorty NP	Cover Screw Brånemark System® NP	Implant Driver NP Brånemark System® Implant Driver Wrench Adapter NP	Abutment Screw Multi-unit Angled Prosthetic Screw Multi-unit	Healing Abutment NP	Immediate Temporary Abutment NP	Procera Esthetic Abutment NP	Impression Coping Closed Tray NP
NobelSpeedy® Groovy NP					Plastic Coping Immediate NP	Esthetic Abutment NP	Impression Coping Open Tray NP
					Temporary Abutment NP	15° Esthetic Abutment NP	Impression Coping Open Tray Multi-unit NP
					QuickTemp™ Abutment Conical NP	Snappy™ Abutment NP	
					Plastic Coping QuickTemp™	Universal Base NP	Impression Coping Closed Tray Multi-unit NP
					Abutment Conical NP	Universal Base Burn-out Coping NP	
					Temporary Abutment Engaging NP	GoldAdapt™ Engaging NP	Impression Coping Bar Closed Tray Multi-unit NP
					Temporary Abutment Non-Engaging NP	GoldAdapt™ Non-Engaging NP	Temporary Coping Multi-unit Titanium NP
					Temporary Abutment Plastic Engaging NP	Multi-Unit Abutment NP	Temporary Coping Multi-unit Plastic NP
					Temporary Abutment Plastic Non-Engaging NP	17° Multi-Unit Abutment NP	
						Locator Abutment NP	
						Gold Coping Bar Multi-unit (Prosthetic screw included) NP	
NobelSpeedy® Shorty RP	Cover Screw Brånemark System® RP	Implant Driver RP Brånemark System Implant Driver Wrench Adapter RP		Healing Abutment RP	Immediate Temporary Abutment RP	Procera Esthetic Abutment RP	Impression Coping Closed Tray RP
NobelSpeedy® Groovy RP					Plastic Coping Immediate Temporary Abutment RP	Esthetic Abutment RP	Impression Coping Open Tray RP
					QuickTemp™ Abutment Conical RP	15° Esthetic Abutment RP	Impression Coping Bar Brånemark System® RP
					Plastic Coping QuickTemp™	Snappy™ Abutment RP	
					Abutment Conical RP	Universal Base RP	Impression Coping Open Tray Multi-unit RP
					Plastic Coping QuickTemp™	Universal Base Burn-out Coping RP	
					Abutment Conical RP	Universal Base Burn-out Coping RP	Impression Coping Closed Tray Multi-unit RP
					Temporary Abutment Engaging RP	GoldAdapt™ Engaging RP	Impression Coping Bar Closed Tray Multi-unit RP
					Temporary Abutment Non-Engaging RP	GoldAdapt™ Non-Engaging RP	
					Temporary Abutment Plastic Engaging RP	Multi-Unit Abutment RP	Impression Coping Bar Closed Tray Multi-unit RP
					Temporary Abutment Plastic Non-Engaging RP	17° Multi-Unit Abutment RP	Temporary Coping Multi-unit Titanium RP
						30° Multi-Unit Abutment RP	Temporary Coping Multi-unit Plastic RP
						30° Multi-Unit Abutment Non-Engaging RP	
						Locator Abutment RP	
						Ball Abutment Titanium RP	
						Gold Abutment Bar Implant Level (vite clinica inclusa) RP	
						Gold Coping Bar Multi-unit (vite protetica inclusa) RP	
NobelSpeedy® Shorty WP	Cover Screw Brånemark System® WP	Implant Driver WP Brånemark System Implant Driver Wrench Adapter WP	Prosthetic Screw Multi-unit	Healing Abutment WP	Immediate Temporary Abutment WP	Procera Esthetic Abutment WP	Impression Coping Closed Tray WP
NobelSpeedy® Groovy WP					Plastic Coping Immediate Temporary Abutment WP	Esthetic Abutment WP	Impression Coping Open Tray WP
					QuickTemp™ Abutment Conical WP	15° Esthetic Abutment WP	Impression Coping Open Tray Multi-unit WP
					Plastic Coping QuickTemp™	Snappy™ Abutment WP	
					Abutment Conical WP	Universal Base WP	Impression Coping Closed Tray Multi-unit WP
					Plastic Coping QuickTemp™	Universal Base Burn-out Coping WP	
					Abutment Conical WP	Universal Base Burn-out Coping WP	Impression Coping Bar Closed Tray Multi-unit WP
					Temporary Abutment Engaging WP	GoldAdapt™ Engaging WP	Temporary Coping Multi-unit Titanium WP
					Temporary Abutment Non-Engaging WP	GoldAdapt™ Non-Engaging WP	
					Temporary Abutment Plastic Engaging WP	Multi-Unit Abutment WP	Temporary Coping Multi-unit Plastic WP
					Temporary Abutment Plastic Non-Engaging WP	Locator Abutment WP	

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IFU1008. NobelReplace® Select™ TC

Nome prodotto	Vite di copertura	Driver per impianto	Viti cliniche	Abutment			Transfer per impronta	Altro
				Guarigione	Provvisorio	Permanente		
Replace Select™ TC NP	Healing Screw RplSel TC NP Cover Screw NobRpl NP	Implant Driver NobRpl NP	Screw Multi-unit Angled Abutm NobRpl NP Screw Ceramic Abutment NobRpl NP Abutment Screw NobRpl NP	Healing Abutment NobRpl NP	Temporary Abutment Engaging NobRpl NP Temporary Abutment Non-Eng NobRpl NP Temporary Abutment Plastic Eng NobRpl NP Temp Abutm Plastic Non-Eng NobRpl NP Immediate Temporary Abutment NobRpl NP	Snappy™ Abutment NobRpl NP Esthetic Abutment NobRpl NP 15° Esthetic Abutment NobRpl NP Universal Base Tri-Channel NP GoldAdapt™ Engaging NobRpl NP GoldAdapt™ Non-Engaging NobRpl NP Multi-unit Abutment NobRpl NP 17° Multi-unit Abutment NobRpl NP Ball Abutment Titanium NobRpl NP NobelProcera® Ab Ti NobelReplace® NP w-u NobelProcera® Ab Ti NobelReplace® NP Procera® Abutm Zr NobRpl NP	Impr Coping Open Tray NobRpl NP Ø 4,5 mm Impr Coping Closed Tray NobRpl NP Impr Cop Cl Tray Low Prof NRpl NP Impr Coping Open Tray NobRpl NP	PIB IBO Impl Retr Instr Hex & Tri-Ch NP/RP 22 mm Bone Mill with Guide NobRpl NP Ø 4,6 Implant Rescue Collar Tri-Channel Ø 3,5
Replace Select™ TC RP	Healing Screw RplSel TC RP Cover Screw NobRpl RP	Implant Driver NobRpl RP	Screw Multi-unit Angled Abutm NobRpl RP Screw Ceramic Abutment NobRpl RP/WP/6.0 Abutment Screw NobRpl RP/WP/6.0	Healing Abutment NobRpl RP	Temporary Abutment Engaging NobRpl RP Temporary Abutment Non-Eng NobRpl RP Temporary Abutment Plastic Eng NobRpl RP Temp Abutm Plastic Non-Eng NobRpl RP Immediate Temporary Abutment NobRpl RP	Snappy™ Abutment NobRpl RP Esthetic Abutment NobRpl RP 15° Esthetic Abutment NobRpl RP Universal Base Tri-Channel RP GoldAdapt™ Engaging NobRpl RP GoldAdapt™ Non-Engaging NobRpl RP Multi-unit Abutment NobRpl RP 30° Multi-unit Abutment NobRpl RP 30° Multi-u Abutm Non-Eng NobRpl RP 17° Multi-unit Abutment NobRpl RP Ball Abutment Titanium NobRpl NP NobelProcera® Ab Ti NobelReplace® RP w-u NobelProcera® Ab Ti NobelReplace® RP Procera® Abutm Zr NobRpl RP	Impr Coping Closed Tray NobRpl RP Impr Cop Cl Tray Low Prof NRpl RP Impr Coping Open Tray NobRpl RP	PIB IBO Impl Retr Instr Hex & Tri-Ch NP/RP 31 mm Bone Mill with Guide NobRpl RP Ø 5,3 Implant Rescue Collar Tri-Channel Ø 4,3

IFU1010. NobelReplace® Conical Connection, NobelReplace® Conical Connection PMC, NobelReplace® Conical Connection TiUltra™

Nome prodotto	Vite di copertura	Driver per impianto	Viti cliniche	Abutment			Transfer per impronta	Altro
				Guarigione	Provvisorio	Permanente		
NobelReplace® Conical Connection NP	Cover Screw CC NP	Implant driver CC NP	Clinical Screw CC NP	Healing abutment CC NP	Slim Temporary Abutment CC NP	Universal Base CC NP	Open Tray CC NP	Guided Implant Mount NobRpl CC NP
NobelReplace® Conical Connection PMC NP			Omnigrip™ Clinical Screw CC NP	Healing abutment bridge NP	Temporary Snap Abutment CC NP	Esthetic Abutment CC NP	Closed Tray CC NP	Bone Mill Guide CC NP
NobelReplace® Conical Connection TiUltra™ NP			On1™ Clinical Screw NP Screw MU Angled Abutment CC NP	Slim healing abutment CC NP	Temporary Abutment Non Engaging CC NP	Procera Esthetic Abutment CC NP Multi-Unit Abutment Plus CC NP (0°/17°/30°) On1™ Base NP	Bridge Open Tray CC NP	Guide Pin CC NP NobelProcera®: NobelProcera® Abutment Zirconia CC NP NobelProcera® Abutment Ti CC NP NobelProcera® FCZ Implant Crown ASC CC NP Implant Bridge Restorations (Ti, Zr) Fixed e Fixed Removable Bars (Ti) Metal Adapter CC NP
NobelReplace® Conical Connection RP	Cover Screw CC RP	Implant driver CC RP	Clinical Screw CC RP/WP	Healing abutment CC RP	Slim Temporary Abutment CC RP	Universal Base CC NP	Open Tray CC RP	Guided Implant Mount NobRpl CC RP
NobelReplace® Conical Connection PMC RP			Omnigrip™ Clinical Screw CC RP/WP	Healing abutment bridge RP	Temporary Snap Abutment CC RP	Esthetic Abutment CC RP	Closed Tray CC RP	Bone Mill Guide CC RP
NobelReplace® Conical Connection TiUltra™ RP			On1™ Clinical Screw RP Screw MU Angled Abutment CC RP	Slim healing abutment CC RP	Temporary Abutment Non Engaging CC RP	Procera Esthetic Abutment CC RP Multi-Unit Abutment Plus CC RP (0°/17°/30°) On1™ Base RP	Bridge Open Tray CC RP	Guide Pin CC RP NobelProcera®: NobelProcera® ASC Abutment Zirconia CC RP NobelProcera® Abutment Ti CC RP NobelProcera® FCZ Implant Crown ASC CC RP Implant Bridge Restorations (Ti, Zr) Fixed e Fixed Removable Bars (Ti) Metal Adapter CC RP

IFU1012. Replace Select™ Tapered TiUnite®, Replace Select™ Tapered PMC

Nome prodotto	Vite di copertura	Driver per impianto	Viti cliniche	Abutment			Transfer per impronta	Altro
				Guarigione	Provvisorio	Permanente		
Replace Select™ Tapered NP	Cover Screw NobRpl NP Healing Screw RplSel TC NP	Implant Driver NobRpl NP	Abutment Screw NobelReplace® NP Screw Multi-unit Angled Abutment NobRpl NP Screw Ceramic Abutment NobRpl NP	Healing Abutment NobRpl NP	Temporary Abutment Engaging NobRpl NP Temporary Abutment Non-Eng NobRpl NP Immediate Temporary Abutment NobRpl NP Plastic/Temp Coping Esthetic Abutment NP Plastic/Temp Coping 15° Esthetic Ab NP	Snappy Abutment NobRpl NP Esthetic Abutment NobRpl NP 15° Esthetic Abutment NobRpl NP Universal Base Tri-Channel NP Universal Base Burn-out Coping NP/RP GoldAdapt™ Engaging NobRpl NP GoldAdapt™ Non-Engaging NobRpl NP Multi-unit Abutment NobRpl NP 17° Multi-unit Abutment NobRpl NP Locator Abut for 3.5 RPL Sel Impl. Ball Abutment Titanium NobRpl NP NobelProcera® Ab Ti NobelReplace® NP Procera® Abutment Zr NobRpl NP	Impr Coping Open Tray NobRpl NP	PIB IBO Bone Mill with Guide NobRpl NP Bone Mill Guide NobelReplace® NP Impl Retr Instr Hex & Tri-Ch NP/RP Implant Rescue Collar Tri-Channel Ø 3,5
Replace Select™ Tapered RP	Cover Screw NobRpl RP Healing Screw RplSel TC NP	Implant Driver NobRpl RP	Abutment Screw NobRpl RP/WP/6.0 Screw Multi-unit Angled Abutment NobRpl RP Screw Ceramic Abutment NobRpl RP/WP/6.0	Healing Abutment NobRpl RP	Temporary Abutment Engaging NobRpl RP Temporary Abutment Non-Eng NobRpl RP Immediate Temporary Abutment NobRpl RP	Snappy Abutment NobRpl RP Esthetic Abutment NobRpl RP 15° Esthetic Abutment NobRpl RP Universal Base Tri-Channel RP Universal Base Burn-out Coping NP/RP GoldAdapt™ Engaging NobRpl RP GoldAdapt™ Non-Engaging NobRpl RP Multi-unit Abutment NobRpl RP 17° Multi-unit Abutment NobRpl RP 30° Multi-unit Abutment NobRpl RP Locator Abut for 4.3 RPL Sel Impl. Ball Abutment Titanium NobRpl RP 3 mm NobelProcera® Ab Ti NobelReplace® RP Procera® Abutment Zr NobRpl RP	Impl Retr Instr Hex & Tri-Ch NP/RP	PIB IBO Bone Mill Guide NobelReplace® RP Bone Mill with Guide NobRpl RP Impl Retr Instr Hex & Tri-Ch NP/RP Implant Rescue Collar Tri-Channel Ø 4,3

Nome prodotto	Vite di copertura	Driver per impianto	Viti cliniche	Abutment			Transfer per impronta	Altro
				Guarigione	Provvisorio	Permanente		
Replace Select™ Tapered WP	Cover Screw NobRpl WP	Implant Driver NobRpl WP	Screw Ceramic Abutment NobRpl RP/ WP/6.0	Healing Abutment NobRpl WP	Temporary Abutment Engaging NobRpl WP Temporary Abutment Non-Eng NobRpl WP Temp Abutment Plastic Non-Eng NobRpl WP Immediate Temporary Abutment NobRpl WP Plastic/Temp Cop Esthetic Ab NobRpl WP Pl/Temp Coping 15° Esthetic Ab NobRpl WP	Snappy Abutment NobRpl WP Esthetic Abutment NobRpl WP 15° Esthetic Abutment NobRpl WP Universal Base Burn-out Coping WP Universal Base Tri-Channel WP GoldAdapt™ Non-Engaging NobRpl WP GoldAdapt™ Engaging NobRpl WP Multi-unit Abutment NobRpl WP Locator Abut for RPL Sel Impl. NobelProcera® Ab Ti NobelReplace® WP Procera® Abutment Zr NobRpl WP	Impr Coping Open Tray NobRpl WP	PIB IBO Bone Mill Guide NobelReplace® WP Bone Mill with Guide NobRpl WP Ø 6,5 Impl Retr Instr CC RP & Tri-Ch WP Impl Retr Instr CC RP& Tri-Ch WP& TCC RP
Replace Select™ Tapered 6.0	Cover Screw NobRpl 6.0 mm	Implant Driver NobRpl 6.0	Screw Ceramic Abutment NobRpl RP/ WP/6.0	Healing Abutment NobRpl 6.0	Temporary Abut Plastic Eng NobRpl 6.0 Temp Abutment Plastic Non-Eng NobRpl 6.0 Temporary Abutment Engaging NobRpl 6.0 Temporary Abutment Non-Eng NobRpl 6.0 Immediate Temporary Abutment NobRpl 6.0	Snappy Abutment NobRpl 6.0 Esthetic Abutment NobRpl 6.0 15° Esthetic Abutment NobRpl 6.0 GoldAdapt™ Non-Engaging NobRpl 6.0 GoldAdapt™ Engaging NobRpl 6.0 NobelProcera® Ab Ti NobelReplace® 6.0 Procera® Abutment Zr NobRpl 6.0	Impr Coping Open Tray NobRpl 6.0	PIB IBO Bone Mill Guide NobRpl 6.0 Bone Mill with Guide NobRpl 6.0 Ø 7 Impl Retr Instr Tri-Channel 6.0 22 mm

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Vite di copertura	Tipo di connessione	Piattaforma implantare	Impianto compatibile	Cacciavite
Cover Screw Brånemark System® NP	Esagono esterno	NP	Brånemark System® NobelSpeedy® Groovy	Cover Screw Driver Brånemark System Hexagon (IFU1085)
Cover Screw Brånemark System® RP		RP	Brånemark System® NobelSpeedy® Groovy NobelZygoma™ 0°	
Cover Screw Brånemark System® WP		WP	Brånemark System® NobelSpeedy® Groovy	
Cover Screw Conical Connection 3.0	Connessione conica (CC)	3.0	Impianti Nobel Biocare dotati di connessione conica (CC).	Unigrip™ (IFU1085)
Cover Screw Conical Connection NP		NP	Impianti Nobel Biocare dotati di connessione conica (CC).	
Cover Screw Conical Connection RP		RP	Impianti Nobel Biocare dotati di connessione conica (CC).	
Cover Screw Conical Connection WP		WP	Impianti Nobel Biocare dotati di connessione conica (CC).	
Cover Screw Nobel Biocare N1™ TCC NP	Connessione conica triviale (TCC)	NP	Nobel Biocare N1™ TiUltra™ TCC	Omnigrip™ Mini (IFU1085)
Cover Screw Nobel Biocare N1™ TCC RP		RP	Nobel Biocare N1™ TiUltra™ TCC	
Cover Screw NobelReplace® NP	Interna trilobata	NP	NobelReplace® Tapered Groovy, ReplaceSelect™	Unigrip™ (IFU1085)
Cover Screw NobelReplace® RP		RP		
Cover Screw NobelReplace® WP		WP		
Cover Screw NobelReplace® 6.0 mm		6.0		

IFU1018. Esthetic Abutment Conical Connection/Bmk system®/ NobelReplace™, Esthetic Abutment Nobel Biocare N1™ TCC

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Esthetic Abutment	Tipo di connessione	Vite clinica	Codifica a colori	Cacciavite	Cappetta in plastica/provisoria
Esthetic Abutment Conical Connection 3.0	Connessione conica	Clinical Screw Conical Connection 3.0		Unigrip™	-
Esthetic Abutment Conical Connection NP	Connessione conica	Clinical Screw Conical Connection NP	●	Unigrip™	-
Esthetic Abutment Conical Connection RP	Connessione conica	Clinical Screw Conical Connection RP/WP	●	Unigrip™	-
Esthetic Abutment Conical Connection WP	Connessione conica	Clinical Screw Conical Connection RP/WP	●	Unigrip™	-
Esthetic Abutment Bmk System NP	Esagonale esterna	Abutment Screw Brånemark System® NP		Unigrip™	Plastic/Temporary Coping Esthetic Abutment
Esthetic Abutment Bmk System RP	Esagonale esterna	Abutment Screw Brånemark System® RP		Unigrip™	Plastic/Temporary Coping Esthetic Abutment
Esthetic Abutment Bmk System WP	Esagonale esterna	Abutment Screw Brånemark System® WP		Unigrip™	Plastic/Temporary Coping Esthetic Abutment
Esthetic Abutment NobelReplace® NP	Trilobata	Abutment Screw NobelReplace® NP	●	Unigrip™	Plastic/Temporary Coping Esthetic Abutment

Esthetic Abutment	Tipo di connessione	Vite clinica	Codifica a colori	Cacciavite	Cappetta in plastica/provisoria
Esthetic Abutment NobelReplace® RP	Trilobata	Abutment Screwtrub NobelReplace® RP/WP/6.0	●	Unigrip™	Plastic/Temporary Coping Esthetic Abutment
Esthetic Abutment NobelReplace® WP	Trilobata	Abutment Screwtrub NobelReplace® RP/WP/6.0	●	Unigrip™	Plastic/Temporary Coping Esthetic Abutment
Esthetic Abutment NobelReplace® 6.0	Trilobata	Abutment Screwtrub NobelReplace® RP/WP/6.0	●	Unigrip™	Plastic/Temporary Coping Esthetic Abutment
Esthetic Abutment Nobel Biocare N1™ TCC NP	Connessione conica triviale	Clinical Screw Nobel Biocare N1 TCC NP	● (vite)	Omnigrip™ Mini	-
Esthetic Abutment Nobel Biocare N1™ TCC RP	Connessione conica triviale	Clinical Screw Nobel Biocare N1 TCC RP	● (vite)	Omnigrip™ Mini	-

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IFU1040. Abutment Release Pins CC

Abutment	Abutment Release Pin CC 3.0	Abutment Release Pin CC NP	Abutment Release Pin CC RP/WP
Temporary Abutment Engaging Conical Connection	✓	n/d	n/d
Esthetic Abutment Conical Connection	✓	n/d	n/d
GoldAdapt™ Engaging Conical Connection	✓	n/d	n/d
NobelProcera® Abutment Zirconia Conical Connection	n/d	✓	✓

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IFU1048. Frese Tapered Drill e Dense Bone Drill, maschiatori Screw Tap

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Vite clinica/ dell'abutment/ protesica	Abutment/Manufatto	Cacciavite
Omnigrip™ Clinical Screw CC	NobelProcera® Angulated Screw Channel Abutment Zr CC NobelProcera® HT ML FCZ Implant Crown CC NobelProcera® HT ML FCZ Implant Bridge CC NobelProcera® Zirconia Implant Bridge CC	Omnigrip™
Clinical Screw Nobel Biocare N1™ TCC	Universal Abutment TCC 17°/30° Multi-unit Abutment Xeal™ TCC Temporary Abutment TCC Healing Abutment TCC Esthetic Abutment Nobel Biocare N1™ TCC Blank per abutment in titanio Nobel Biocare N1™ TCC	Omnigrip™ Mini
Abutment Screw NobelReplace®	Universal Base Tri-channel Esthetic Abutment NobelReplace® Temporary Abutment NobelReplace® NobelProcera® Abutment Ti NobelReplace® NobelProcera® Implant Bridge Ti NobelReplace® NobelProcera® Implant Bar Overdenture NobelReplace® Snappy™ Abutment NobelReplace® GoldAdapt™ NobelReplace® Gold Abutment Bar NobelReplace® Narrow Profile Abutment NobelReplace®	Unigrip™
Screw Ceramic Abutment NobelReplace®	NobelProcera® Abutment Zr NobelReplace® NobelProcera® HT ML FCZ Implant Bridge NobelReplace® NobelProcera® Zirconia Implant Bridge NobelReplace® Procera® Esthetic Abutment NobelReplace®	Unigrip™
Abutment Screw Brånemark System®	Universal Base External Hex Esthetic Abutment Brånemark System® Temporary Abutment Brånemark System® NobelProcera® Abutment Ti Brånemark System® NobelProcera® Implant Bridge Ti Brånemark System® NobelProcera® Implant Bar Overdenture Brånemark System® Snappy™ Abutment Brånemark System® GoldAdapt™ Brånemark System® Gold Abutment Brånemark System® CeraOne Abutment Brånemark System®	Unigrip™
Screw Ceramic Abutment Brånemark System®	NobelProcera® Abutment Zr Brånemark System® NobelProcera® HT ML FCZ Implant Bridge Brånemark System® NobelProcera® Zirconia Implant Bridge Brånemark System® Procera® Esthetic Abutment Brånemark System®	Unigrip™
Screw Multi-unit Angled Abutment CC	17°/30° Multi-unit Abutment CC	Unigrip™
Vite clinica Multi-unit Abutment Nobel Biocare N1™ TCC	Straight Multi-unit Abutment Xeal™ TCC	Multi-unit
Screw Multi-unit Angled Abutment NobelReplace®	17°/30° Multi-unit Abutment NobelReplace®	Unigrip™
Screw Multi-unit Angled Abutment Brånemark System®	17°/30° Multi-unit Abutment Brånemark System®	Unigrip™

Vite clinica/ dell'abutment/ protesica	Abutment/Manufatto	Cacciavite
Vite protesica Multi-unit Abutment	Temporary Coping Multi-unit Abutment NobelProcera® Implant Bridge Ti Multi-unit NobelProcera® Implant Bridge Zr Multi-unit NobelProcera® Implant Bar Overdenture Multi-unit NobelProcera® HT ML FCZ Implant Bridge Multi-unit Gold Coping Multi-unit	Unigrip™
Prosthetic Screw Multi-unit Abutment Omnigrip™ Mini	NobelProcera® Zirconia Implant Bridge Multi-unit	Omnigrip™ Mini
NobelZygoma 0° Angled Multi-unit Abutment Screw	45°/60° Multi-unit Abutment External Hex	Unigrip™
Brånemark System® Zygoma Abutment Screw	NobelProcera® Implant Bridge Zygoma	Unigrip™
Brånemark System® Zygoma Angled Multi-unit Abut Screw	17° Multi-unit Abut Brånemark System® Zygoma	Unigrip™
Omnigrip™ Clinical Screw Titanium (CC & Tri-Channel)	Nobel Procera® Titanium Abutment ASC (CC & Tri-Channel)	Omnigrip™
Temporary Prosthetic Screw Multi-unit Abutment	Multi-unit Abutment Plus CC NP/RP/WP Multi-unit Abutment Xeal™ CC NP/RP/WP 17° Multi-unit Abutment Plus CC NP/RP/WP 17° Multi-unit Abutment Xeal™ CC NP/RP/WP 30° Multi-unit Abutment Plus CC NP/RP 30° Multi-unit Abutment Xeal™ CC NP/RP Multi-unit Abutment Xeal™ Nobel Biocare N1™ TCC NP/RP 17° Multi-unit Abutment Xeal™ Nobel Biocare N1™ TCC NP/RP 30° Multi-unit Abutment Xeal™ Nobel Biocare N1™ TCC RP Multi-unit Abutment NobRpl NP/RP/WP 17° Multi-Unit Abutment NobelReplace® NP/RP 30° Multi-Unit Abutment NobelReplace® RP Multi-unit Abutment Brånemark System® NP/RP 17° Multi-unit Abutment Brånemark System® NP/RP 30° Multi-Unit Abutment Brånemark System® RP Multi-unit Abutment Brånemark System® Zygoma 17° Multi-unit Abutment Brånemark System® Zygoma 45° Multi-unit Abutment External Hex RP 60° Multi-unit Abutment External Hex RP Multi-unit Abutment Replica Analoghi Elos Accurate Nobel Biocare ReverseScan Multi-unit	Omnigrip™ Mini

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IFU1074. Concetto On1™

Nome prodotto	Impianto compatibile	Abutment	Vite	Driver
On1™ Base CC NP 1.75 mm	Impianti Nobel Biocare dotati di connessione conica (CC).	On1™ Temporary Abutment Engaging	On1™ Clinical Screw 1.75 mm NP	On1™ Screwdriver
On1™ Base CC NP 2.5 mm		On1™ Temporary Abutment Non-Engaging	On1™ Clinical Screw 2.5 mm NP	
On1™ Base CC RP 1.75 mm		On1™ Universal Abutment	On1™ Clinical Screw 1.75 mm RP	
On1™ Base CC RP 2.5 mm		On1™ Universal Abutment Non-Engaging	On1™ Clinical Screw 2.5 mm RP	
On1™ Base CC WP 1.75 mm		On1™ Esthetic Abutment	On1™ Clinical Screw 1.75 mm WP	
On1™ Base CC WP 2.5 mm			On1™ Clinical Screw 2.5 mm WP	
On1™ Universal Abutment 0.3 mm NP	N/D	On1™ Base	On1™ Prosthetic Screw	Unigrip™ Screwdriver
On1™ Universal Abutment 1.25 mm NP		On1™ Base Xeal	On1™ Prosthetic Lab Screw	
On1™ Universal Abutment 0.3 mm RP				
On1™ Universal Abutment 1.25 mm RP				
On1™ Universal Abutment 0.3 mm WP				
On1™ Universal Abutment 1.25 mm WP				
On1™ Esthetic Abutment Titanium 0.3 mm NP	N/D	On1™ Base	On1™ Prosthetic Screw	Unigrip™ Screwdriver
On1™ Esthetic Abutment Titanium 0.3 mm RP		On1™ Base Xeal	On1™ Prosthetic Lab Screw	
On1™ Esthetic Abutment Titanium 0.3 mm WP				
On1™ Base Xeal™ CC NP 1.75 mm	Impianti Nobel Biocare dotati di connessione conica (CC).	On1™ Temporary Abutment Engaging	On1™ Clinical Screw 1.75 mm NP	On1™ Screwdriver
On1™ Base Xeal™ CC NP 2.5 mm		On1™ Temporary Abutment Non-Engaging	On1™ Clinical Screw 2.5 mm NP	
On1™ Base Xeal™ CC RP 1.75 mm		On1™ Universal Abutment	On1™ Clinical Screw 1.75 mm RP	
On1™ Base Xeal™ CC RP 2.5 mm		On1™ Universal Abutment Non-Engaging	On1™ Clinical Screw 2.5 mm RP	
On1™ Base Xeal™ CC WP 1.75 mm		On1™ Esthetic Abutment	On1™ Clinical Screw 1.75 mm WP	
On1™ Base Xeal™ CC WP 2.5 mm			On1™ Clinical Screw 2.5 mm WP	
On1™ Universal Abutment Non-Eng NP 0.3 mm	N/D	On1™ Base	On1™ Prosthetic Screw	Unigrip™ Screwdriver
On1™ Universal Abutment Non-Eng NP 1.25 mm		On1™ Base Xeal	On1™ Prosthetic Lab Screw	
On1™ Universal Abutment Non-Eng RP 0.3 mm				
On1™ Universal Abutment Non-Eng RP 1.25 mm				
On1™ Universal Abutment Non-Eng WP 0.3 mm				
On1™ Universal Abutment Non-Eng WP 1.25 mm				
On1™ Healing Cap NP 1.5 mm	N/D	On1™ Base	On1™ Prosthetic Screw	Unigrip™ Screwdriver
On1™ Healing Cap NP 2.5 mm		On1™ Base Xeal		
On1™ Healing Cap RP 1.5 mm				
On1™ Healing Cap RP 2.5 mm				
On1™ Healing Cap WP 1.5 mm				
On1™ Healing Cap WP 2.5 mm				
On1™ IOS Healing Cap 6 mm NP	N/D	On1™ Base	On1™ Prosthetic Screw	Unigrip™ Screwdriver
On1™ IOS Healing Cap 4.5 mm NP		On1™ Base Xeal		
On1™ IOS Healing Cap 6 mm RP				
On1™ IOS Healing Cap 4.5 mm RP				
On1™ IOS Healing Cap 5 mm WP				
On1™ IOS Healing Cap 4 mm WP				
OOn1™ Temporary Abutment Engaging NP	N/D	On1™ Base	On1™ Prosthetic Screw	Unigrip™ Screwdriver
On1™ Temporary Abutment Non-Engaging NP		On1™ Base Xeal	On1™ Prosthetic Lab Screw	
On1™ Temporary Abutment Engaging RP				
On1™ Temporary Abutment Non-Engaging RP				
On1™ Temporary Abutment Engaging WP				
On1™ Temporary Abutment Non-Engaging WP				

Nome prodotto	Impianto compatibile	Abutment	Vite	Driver
On1™ Impression Coping Closed Tray NP	N/D	On1™ Base On1™ Base Xeal	N/D	Unigrip™ Screwdriver
On1™ Impression Coping Closed Tray RP				
On1™ Impression Coping Closed Tray WP				
On1™ Impression Coping Open Tray NP				
On1™ Impression Coping Open Tray RP				
On1™ Impression Coping Open Tray WP				
On1™ Impression Coping Open Tray N-Eng NP	N/D	On1™ Base On1™ Base Xeal	N/D	Unigrip™ Screwdriver
On1™ Impression Coping Open Tray N-Eng RP				
On1™ Impression Coping Open Tray N-Eng WP				
On1™ Clinical Screw 1.75 mm NP	Impianti Nobel Biocare dotati di connessione conica (CC).	On1™ Temporary Abutment Engaging On1™ Temporary Abutment Non-Engaging On1™ Universal Abutment On1™ Universal Abutment Non-Engaging On1™ Esthetic Abutment	On1™ Prosthetic Screw On1™ Prosthetic Lab Screw On1™ Impression Coping Screw	On1™ Screwdriver
On1™ Clinical Screw 2.5 mm NP				
On1™ Clinical Screw 1.75 mm RP				
On1™ Clinical Screw 2.5 mm RP				
On1™ Clinical Screw 1.75 mm WP				
On1™ Clinical Screw 2.5 mm WP				
On1™ Prosthetic Screw	N/D		On1™ Clinical Screw	Unigrip™ Screwdriver
On1™ Screwdriver Machine 21 mm	N/D	On1™ Base On1™ Base Xeal	On1™ Clinical Screw	N/D
On1™ Screwdriver Manual 25 mm	N/D	On1™ Base On1™ Base Xeal	On1™ Clinical Screw	N/D

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Connection to Handpiece

Connection to Handpiece	Dispositivi compatibili	Impianto
Connection to Handpiece	Guided Implant Mount Brånemark System® NP, RP 3.75, RP 4.0, WP Zygoma Implant Mount	Brånemark System® NobelSpeedy® Zygoma RP
NobelReplace® Connection to Handpiece	Guided Implant Mount NobelActive® NP, RP 4.3, RP 5.0, WP 5.5 Guided Implant Mount NobelParallel™ NP 3.75, RP 4.3, RP 5.0, WP 5.5 Guided Implant Mount NobelReplace® NP 3.5, RP 4.3, WP 5.0, 6.0 Guided Implant Mount NobelReplace® CC NP 3.5, RP 4.3, RP 5.0 Implant Mount Ø 5 mm	Impianti Nobel Biocare dotati di connessione conica (CC).

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Direction Indicator Ø 2/Ø 2.4-2.8 mm e Direction Indicator Tapered NP/RP/WP/6.0

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Handle for Implant Rescue Collar e Drill Guide

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Implant Driver Wrench Adapter Brånemark System® NP/RP/WP

Impianto	Torque Wrench
Brånemark System® NobelSpeedy®	Brånemark System® Manual Torque Wrench

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Nome prodotto	Dispositivi compatibili		Cacciaviti
	Drill Stop/ Drill Stop Kit Box	Frese	
Drill Stop Ø 2	Guided Drill Stop Kit Box Drill Stop Kit Box	Twist Drill con punta 2.0x18-25 mm Guided Twist Drill 2x(10+)-7-13 mm Guided Twist Drill 2x(10+)-7-18 mm Twist Drill con punta 2x7-15 mm Twist Drill 3.2x7-10 mm Twist Drill 1.5x7-15 mm Twist Drill con punta 2x10-18 mm Twist Drill con punta 2x18-25 mm, LS Guided Drill Guide NP per Ø 2 mm Guided Drill Guide RP per Ø 2 mm Guided Drill Guide 6.0/WP per Ø 2 mm	Screwdriver Manual Unigrip™ Screwdriver Machine Unigrip™
Drill Stop Ø 2.8	Guided Drill Stop Kit Box Drill Stop Kit Box	Guided Twist Drill 2.8x(10+)-7-13 mm Guided Twist Drill 2.8x(10+)-7-18 mm Twist Step Drill 2.4/2.8 10-18 mm Twist Step Drill 2.4/2.8 7-15 mm Twist Step Drill 2.4/2.8 7-10 mm Guided Twist Step Drill Ø 2.4/2.8 (10+) 7-13 mm Guided Twist Step Drill Ø 2.4/2.8 (10+) 7-18 mm Twist Step Drill 2.4/2.8x18-25 mm, LS Twist Step Drill 2.4/2.8x18-25 mm Guided Drill Guide NP per Ø 2,8 mm Guided Drill Guide RP per Ø 2,8 mm Guided Drill Guide 6.0/WP per Ø 2,8 mm	Screwdriver Machine Unigrip™ Screwdriver Manual Unigrip™
Drill Stop Ø 3	Guided Drill Stop Kit Box Drill Stop Kit Box	Guided Twist Drill 3x(10+)-7-13 mm Guided Twist Drill 3x(10+)-7-18 mm Twist Drill 3x10-18 mm Twist Drill 3x7-15 mm Twist Drill 3x7-10 mm Guided Drill Guide NP per Ø 3 mm Guided Drill Guide RP per Ø 3 mm Guided Drill Guide 6.0/WP per Ø 3 mm	Screwdriver Machine Unigrip™ Screwdriver Manual Unigrip™
Drill Stop Ø 3.2	Guided Drill Stop Kit Box Drill Stop Kit Box	Twist Step Drill 2.8/3.2 7-10 mm Twist Drill 3.2x18-25 mm Twist Step Drill 2.8/3.2 10-18 mm Twist Step Drill 2.8/3.2 7-15 mm Guided Twist Drill 3.2x(10+)-7-13 mm Twist Drill 3.2x7-10 mm Guided Twist Drill 3.2x(10+)-7-18 mm Twist Drill 3.2x10-18 mm Guided Twist Step Drill Ø 2.8/3.2 (10+) 7-18 mm Guided Twist Step Drill Ø 2.8/3.2 (10+) 7-13 mm Twist Drill 3.2x7-15 mm Guided Drill Guide NP per Ø 3.2 mm Guided Drill Guide RP per Ø 3.2 mm	Screwdriver Machine Unigrip™ Screwdriver Manual Unigrip™
Drill Stop Ø 3.4	Guided Drill Stop Kit Box Drill Stop Kit Box	Guided Twist Drill 3.4x(10+)-7-13 mm Guided Twist Drill 3.4x(10+)-7-18 mm Twist Drill 3.4x10-18 mm Twist Drill 3.4x7-15 mm Twist Drill 3.4x7-10 mm Twist Drill 3.4x18-25 mm Guided Drill Guide RP per Ø 3,4 mm	Screwdriver Machine Unigrip™ Screwdriver Manual Unigrip™
Drill Stop Ø 3.6	Guided Drill Stop Kit Box Drill Stop Kit Box	Twist Step Drill 3.2/3.6 7-15 mm Twist Step Drill 3.2/3.6 7-10 mm Guided Twist Step Drill Ø 3.2/3.6 (10+) 7-18 mm Guided Twist Step Drill Ø 3.2/3.6 (10+) 7-13 mm Twist Step Drill 3.2/3.6 10-18 mm Guided Drill Guide RP per Ø 3.6 mm Guided Drill Guide WP/6.0 per Ø 3.6 mm	Screwdriver Machine Unigrip™ Screwdriver Manual Unigrip™

Nome prodotto	Dispositivi compatibili		Cacciaviiti
	Drill Stop/ Drill Stop Kit Box	Frese	
Drill Stop Ø 3.8	Guided Drill Stop Kit Box Drill Stop Kit Box	Guided Twist Drill 3.8x(10+)-7-13 mm Guided Twist Drill 3.8x(10+)-7-18 mm Guided Drill Guide 6.0/WP per Ø 3.8 mm	Screwdriver Machine Unigrip™ Screwdriver Manual Unigrip™
Drill Stop Ø 4.2	Guided Drill Stop Kit Box Drill Stop Kit Box	Guided Twist Step Drill 3.8/4.2 (10+) 7-18 Twist Step Drill 3.8/4.2 10-18 mm Twist Step Drill 3.8/4.2 7-15 mm Twist Step Drill 3.8/4.2 7-10 mm Guided Twist Drill 4.2x(10+)-7-13 mm Guided Twist Drill 4.2x(10+)-7-18 mm Guided Twist Step Drill 3.8/4.2 (10+) 7-13 Guided Drill Guide RP per Ø 4.2 mm Guided Drill Guide 6.0/WP per Ø 4.2 mm	Screwdriver Machine Unigrip™ Screwdriver Manual Unigrip™
Drill Stop Kit Box	Drill Stop Ø 2 Drill Stop Ø 2.8 Drill Stop Ø 3 Drill Stop Ø 3.2 Drill Stop Ø 3.4 Drill Stop Ø 3.6 Drill Stop Ø 3.8 Drill Stop Ø 4.2	Twist Drill con punta 2.0x18-25 mm Twist Drill con punta 2x7-15 mm Twist Drill 3.2x7-10 mm Twist Drill 1.5x7-15 mm Twist Drill con punta 2x10-18 mm Twist Drill con punta 2x18-25 mm, LS Twist Step Drill 2.4/2.8 10-18 mm Twist Step Drill 2.4/2.8 7-15 mm Twist Step Drill 2.4/2.8 7-10 mm Twist Step Drill 2.4/2.8x18-25 mm, LS Twist Step Drill 2.4/2.8x18-25 mm Twist Step Drill 2.4/2.8x18-25 mm Twist Drill 3x10-18 mm Twist Drill 3x7-15 mm Twist Drill 3x7-10 mm Twist Step Drill 2.8/3.2 7-10 mm Twist Drill 3.2x18-25 mm Twist Step Drill 2.8/3.2 10-18 mm Twist Step Drill 2.8/3.2 7-15 mm Twist Drill 3.2x7-10 mm Twist Drill 3.2x10-18 mm Twist Drill 3.2x7-15 mm Twist Drill 3.4x10-18 mm Twist Drill 3.4x7-15 mm Twist Drill 3.4x7-10 mm Twist Drill 3.4x18-25 mm Twist Step Drill 3.2/3.6 7-15 mm Twist Step Drill 3.2/3.6 7-10 mm Twist Step Drill 3.2/3.6 10-18 mm Twist Step Drill 3.8/4.2 10-18 mm Twist Step Drill 3.8/4.2 7-15 mm Twist Step Drill 3.8/4.2 7-10 mm	N/D

Nome prodotto	Dispositivi compatibili		
	Drill Stop/ Drill Stop Kit Box	Frese	Cacciaviti
Guided Drill Stop Kit Box	Drill Stop Ø 2	Guided Twist Drill 2x(10+)7-13 mm	N/D
	Drill Stop Ø 2.8	Guided Twist Drill 2x(10+)7-18 mm	
	Drill Stop Ø 3	Guided Twist Drill 2.8x(10+)7-13 mm	
		Guided Twist Drill 2.8x(10+)7-18 mm	
	Drill Stop Ø 3.2	Guided Twist Step Drill Ø 2.4/2.8 (10+) 7-13 mm	
	Drill Stop Ø 3.4	Guided Twist Step Drill Ø 2.4/2.8 (10+) 7-18 mm	
		Guided Drill Guide NP per Ø 2.8 mm	
	Drill Stop Ø 3.6	Guided Drill Guide RP per Ø 2.8 mm	
	Drill Stop Ø 3.8	Guided Drill Guide 6.0/WP per Ø 2.8 mm	
		Guided Twist Drill 3x(10+)7-13 mm	
	Drill Stop Ø 4.2	Guided Twist Drill 3x(10+)7-18 mm	
		Guided Twist Drill 3.2x(10+)7-13 mm	
		Guided Twist Drill 3.2x(10+)7-18 mm	
		Guided Twist Step Drill Ø 2.8/3.2 (10+) 7-18 mm	
		Guided Twist Step Drill Ø 2.8/3.2 (10+) 7-13 mm	
		Guided Twist Drill 3.4x(10+)7-13 mm	
		Guided Twist Drill 3.4x(10+)7-18 mm	
		Guided Twist Step Drill Ø 3.2/3.6 (10+) 7-18 mm	
		Guided Twist Step Drill Ø 3.2/3.6 (10+) 7-13 mm	
		Guided Twist Drill 3.8x(10+)7-13 mm	
		Guided Twist Drill 3.8x(10+)7-18 mm	
		Guided Twist Step Drill 3.8/4.2 (10+) 7-18	
		Guided Twist Drill 4.2x(10+)7-13 mm	
		Guided Twist Drill 4.2x(10+)7-18 mm	
		Guided Twist Step Drill 3.8/4.2 (10+) 7-13	

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N. articolo	Nome prodotto	Vite di copertura	Implant Mount e relativa vite	Transfer per impronta e relativo perno guida	Abutment di guarigione	Abutment definitivi diritti e relative viti	Abutment definitivi angolati e relativa vite	Ponti su impianti e Implant Bar Overdenture e relative viti	Zygoma Handle, Connection to handpiece Altro
28862	Zygoma Implant RP 30 mm	Zygoma Implant Cover Screw : 28989 Brånemark System Zygoma Implant Cover Screw : 32424	Zygoma Implant Mount Zygoma Fixture Mount Screw M2specx6,6 mm (il montaimpianto è preassemblato all'impianto)	Brånemark System® Zygoma Impression Coping Open Tray Ø 4 mm Guide Pin Zygoma 20 mm RP	Brånemark System® Zygoma Healing Abutment Ø 4x3 mm Brånemark System® Zygoma Healing Abutment Ø 4x5 mm	Zygoma Abutment Multi-unit RP 3 mm Zygoma Abutment Multi-unit RP 5 mm Brånemark System® Zygoma Multi-unit Abutment 3 mm Brånemark System® Zygoma Multi-unit Abutment 5 mm	Zygoma 17° Abutment Multi-unit RP 2 mm Zygoma 17° Abutment Multi-unit RP 3 mm Brånemark System® Zygoma 17° Multi-unit Abutment 2 mm Brånemark System® Zygoma 17° Multi-unit Abutment 3 mm Brånemark System® Zygoma Angled Multi-unit Abutment Screw	Nobel Procera Implant Bar Overdenture Titanium Procera Implant Bridge Ti livello dell'impianto NobelProcera® Zr Implant Bridge (livello dell'impianto) Brånemark System® Zygoma Abutment Screw	Zygoma Handle Connection to Handpiece Unigrip Screw Driver (alla vite di montaggio dell'impianto)
28863	Zygoma Implant RP 35 mm								
28864	Zygoma Implant RP 40 mm								
28865	Zygoma Implant RP 42.5 mm								
28866	Zygoma Implant RP 45 mm								
28867	Zygoma Implant RP 47.5 mm								
28868	Zygoma Implant RP 50 mm								
28869	Zygoma Implant RP 52.5 mm								

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Strumento	Diametro dell'impianto compatibile
Trephine Drill 3.2/4.0 mm	3,0 mm
Trephine Drill 3.8/4.6 mm	3,3 mm/3,5 mm/3,75 mm
Trephine Drill 4.4/5.2 mm	4,0 mm/4,3 mm
Trephine Drill 5.2/6.2 mm	5,0 mm
Trephine Drill 5.6/6.6 mm	5,5 mm
Trephine Drill 6.2/7.0 mm	6,0 mm

IFU1098. Manual Torque Wrench Surgical e Prosthetic

Chiave	Adattatore per chiave	Strumenti
NobelActive® Manual Torque Wrench Surgical	NOBELREPLACE® Manual Torque Wrench Adapter Surgical	Implant Driver CC 3.0 28 mm (IFU1090) Implant Driver CC 3.0 per Slim Abutment Implant Driver CC NP (IFU1090)
		Implant Driver CC NP per Slim Abutment (IFU1090) Implant Driver CC RP (IFU1090) Implant Driver CC RP per Slim Abutment (IFU1090) Implant Driver CC WP (IFU1090) Strumenti per la rimozione dell'impianto (IFU1097) Abutment Screw Retrieval Instrument (IFU1043)
Brånemark System® Manual Torque Wrench Surgical	Brånemark System® Manual Torque Wrench Surgical	Implant Driver Brånemark Syst NP (IFU1090) Implant Driver Brånemark Syst RP (IFU1090) Implant Driver Brånemark Syst WP (IFU1090)
Manual Torque Wrench Surgical Nobel Biocare N1™	NobelReplace® Manual Torque Wrench Adapter Surgical	Implant Driver Nobel Biocare N1™ TCC NP (IFU1087) Implant Driver Nobel Biocare N1™ TCC RP (IFU1087) Strumenti per la rimozione dell'impianto (IFU1097) Abutment Screw Retrieval Instrument (IFU1043)
Manual Torque Wrench Prosthetic	Manual Torque Wrench Adapter Prosthetic	Screwdriver Machine Unigrip (IFU1085) Screwdriver Machine Multi-Unit (IFU1085) Omnigrip™ Screwdriver Machine (IFU1085) On1™ Base Screwdriver Machine (IFU1074) Screwdriver Machine Multi-unit Brånemark System (IFU1085) Screwdriver Machine Ball Abutment (IFU1085)
		Screwdriver Machine Omnigrip™ Mini (IFU1085) Screwdriver Machine Multi-Unit (IFU1085)

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Abutment originale	Vite sostitutiva	Torque di serraggio	Sostituzione/cacciavite
Standard Abutment RP o EsthetiCone Abutment	Prosthetic Screw Internal Hexagon	10 Ncm	Screwdriver Hexagon 27 mm Machine Screwdriver Hex Long
	Prosthetic Screw Slot		Screwdriver Medium 37 mm Screwdriver Slot Short 27 mm
	Prosthetic Screw Conical		Machine Screwdriver Slot Short Machine Screwdriver Slot Long
Brånemark 3.0 NP Abutment per 3.0 Brånemark Implant	Converter Screw Titanium Unigrip™ fit Ø 3.0	15 Ncm	Unigrip™ Screwdriver*

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NobelSpeedy/Bmk/RepSel TC LiteSet Tray	Bone Mill
	Bone Mill Guide
	Svasatore
	Cover Screw Driver Bmk Syst Hexagon
	Depth Probe
	Direction Indicator Ø 2/Ø 2.4-2.8 mm
	Prolunga per fresa
	Implant Driver Bmk Syst
	Implant Driver NoblRpl
	Implant Driver Wrench Adap Bmk Syst
	Screwdriver Unigrip
	Maschiatori
	Torque Wrench
	Twist Drill

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Prosthetic LiteSet Tray	Handle
	Screwdriver Manual Multi-unit
	Screwdriver NB N1
	Screwdriver Omnigrip
	Screwdriver Omnigrip Mini
	Screwdriver UniGrip
	Torque Wrench

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IFU2009. Guided Pilot Drill Sleeve, Guided Sleeve e Guided Anchor Pin Sleeve

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Nome prodotto	Impianto	Manicotto	Cacciavite	Altro
Guided Surgery Implant Mount NobelActive® NP	NobelActive®/TiUltra™ NP	Guided Sleeve NP	Surgical Driver Screwdriver Manual/ Machine Unigrip™	Manual Torque Wrench Adapter Surgical Connection to Handpiece
Guided Surgery Implant Mount NobelActive® RP 4.3	NobelActive®/TiUltra™ RP 4.3	Guided Sleeve RP		
Guided Surgery Implant Mount NobelActive® RP 5.0	NobelActive®/TiUltra™ RP 5.0	Guided Sleeve 6.0/WP		
Guided Surgery Implant Mount NobelActive® WP 5.5	NobelActive®/TiUltra™ WP 5.5			
Guided Implant Mount Bmk Syst NP	Brånemark System® NP	Guided Sleeve NP	Surgical Driver Screwdriver Manual/ Machine Unigrip™	Brånemark System® Manual Torque Wrench Adapter Surgical Connection to handpiece
Guided Implant Mount Bmk Syst RP	Brånemark System® RP	Guided Sleeve RP		
Guided Implant Mount Bmk Syst WP	Brånemark System® WP	Guided Sleeve 6.0/WP		
Guided Implant Mount NobRpl NP	NobelReplace® NP	Guided Sleeve NP	Surgical Driver Screwdriver Manual/ Machine Unigrip™	NobelReplace® Manual Torque Wrench Adapter Surgical Connection to handpiece
Guided Implant Mount NobRpl RP	NobelReplace® RP	Guided Sleeve RP		
Guided Implant Mount NobRpl WP	NobelReplace® WP	Guided Sleeve 6.0/WP		
Guided Implant Mount NobRpl 6.0	NobelReplace® 6.0	Guided Sleeve 6.0/WP		
Guided Implant Mount NobelReplace® Conical Connection NP 3.5	NobelReplace® CC NP	Guided Sleeve NP	Surgical Driver Screwdriver Manual/ Machine Unigrip™	NobelReplace® Manual Torque Wrench Adapter Surgical Connection to handpiece
Guided Implant Mount NobelReplace® Conical Connection RP 4.3	NobelReplace® CC RP	Guided Sleeve RP		
Guided Implant Mount NobelReplace® Conical Connection RP 5.0	NobelReplace® CC RP 5.0	Guided Sleeve 6.0/WP		
Guided Implant Mount NobelParallel™ CC NP	NobelParallel™ CC NP	Guided Sleeve NP		
Guided Implant Mount NobelParallel™ CC RP 4.3	NobelParallel™ CC RP	Guided Sleeve RP	Surgical Driver Screwdriver Manual/ Machine Unigrip™	NobelReplace® Manual Torque Wrench Adapter Surgical Connection to handpiece
Guided Implant Mount NobelParallel™ CC RP 5.0	NobelParallel™ CC RP 5.0	Guided Sleeve 6.0/WP		
Guided Implant Mount NobelParallel™ CC WP 5.5	NobelParallel™ CC WP	Guided Sleeve 6.0/WP		

Nome prodotto	Impianto	Manicotto	Cacciavite	Altro
Guided Template Abutm w Screw NobelActive® NP	NobelActive®/TiUltra™ NP	Guided Sleeve NP	Screwdriver Manual/ Machine Unigrip™	N/D
Guided Template Abutm w Screw NobelActive® RP 4.3	NobelActive®/TiUltra™ RP 4.3	Guided Sleeve RP		
Guided Template Abutm w Screw NobelActive® RP 5.0	NobelActive®/TiUltra™ RP 5.0	Guided Sleeve 6.0/WP		
Guided Template Abutm w Screw NobRpl 6.0	NobelReplace® CC/ NobelReplace® CC TiUltra™ 6.0 e Replace Select™ Tapered TiUnite® 6.0 implants			
Guided Template Abutm w Screw NobRpl NP	NobelReplace® CC/ NobelReplace® CC TiUltra™ NP e Replace Select™ Tapered TiUnite® NP implants	Guided Sleeve NP		
Guided Template Abutm w Screw NobRpl RP	NobelReplace® CC/ NobelReplace® CC TiUltra™ RP e Replace Select™ Tapered TiUnite® RP implants	Guided Sleeve RP		
Guided Template Abutm w Screw NobRpl WP	NobelReplace® CC/ NobelReplace® CC TiUltra™ WP e Replace Select™ Tapered TiUnite® WP implants	Guided Sleeve 6.0/WP		
Guided Template Abutment CC NP	NobelActive®/TiUltra™ NP, NobelParallel™ CC/ NobelParallel™ CC TiUltra™ NP, NobelReplace® CC/ NobelReplace® CC TiUltra™ NP, NobelReplace® CC PMC NP implants	Guided Sleeve NP		
Guided Template Abutment CC NP	NobelReplace® CC/ NobelReplace® CC TiUltra™ & CC PMC NP implants			
Guided Template Abutment CC RP 4.3	NobelReplace® CC/ NobelReplace® CC TiUltra™ & CC PMC RP 4.3 implants	Guided Sleeve RP		
Guided Template Abutment CC RP 5.0	NobelReplace® CC/ NobelReplace® CC TiUltra™ & CC PMC RP 5.0 implants	Guided Sleeve 6.0/WP		
Guided Template Abutment CC WP 5.5	NobelReplace® CC/ NobelReplace® CC TiUltra™ & CC PMC WP 5.5 implants			
Guided Template Abutment with Screw Brånemark System® NP	Brånemark System® MK III NP and NobelSpeedy® Groovy NP implants	Guided Sleeve NP		
Guided Template Abutment con Screw Brånemark System® RP	Brånemark System® MK III RP and NobelSpeedy® Groovy RP implants	Guided Sleeve RP		
Guided Template Abutment with Screw Brånemark System® WP	Brånemark System® MK III WP and NobelSpeedy® Groovy WP implants	Guided Sleeve 6.0/WP		
Guided Template Abutment with Screw NobelActive® WP 5.5	NobelActive®/TiUltra™ WP 5.5			
Handle for Guided Drill Guide	N/D	N/D	N/D	Guided Drill Guide

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Nome prodotto	Connessioni al manipolo
Guided Start Drill/Counterb Bmk Syst NP	EN ISO 1797
Guided Start Drill/Counterbore NobRpl NP	
Guided Start Drill/Counterb Bmk Syst RP	
Guided Start Drill/Counterbore NobRpl RP	
Guided Start Drill/Counterb BmkSyst WP Ø 5	
Guided Start Drill/Counterbore NobRpl WP	
Guided Start Drill/Counterb BmkSyst WP Ø 6	
Guided Start Drill/Counterb NobRpl 6.0	
Guided Twist Drill 2x(10+)7-18 mm	
Guided Twist Drill 2.8x(10+)7-18 mm	
Guided Twist Drill 3x(10+)7-18 mm	
Guided Twist Drill 3.2x(10+)7-18 mm	
Guided Twist Drill 3.4x(10+)7-18 mm	
Guided Twist Drill 3.8x(10+)7-18 mm	
Guided Twist Drill 4.2x(10+)7-18 mm	
Guided Twist Drill 5x(10+)7-18 mm	
Guided Dense Bone Drill Taper NP 3.5x13 mm	
Guided Dense Bone Drill Taper NP 3.5x16 mm	
Guided Dense Bone Drill Taper RP 4.3x13 mm	
Guided Dense Bone Drill Taper RP 4.3x16 mm	
Guided Dense Bone Drill Taper WP 5x13 mm	
Guided Dense Bone Drill Taper WP 5x16 mm	
Guided Dense Bone Drill Taper 6.0 6x13 mm	
Guided Dense Bone Drill Taper 6.0 6x16 mm	
Guided Screw Tap NP 10-13 mm	
Guided Screw Tap RP Ø 3.75 7-13 mm	
Guided Screw Tap Tapered RP	
Guided Screw Tap WP Ø 5 7-13 mm	
Guided Screw Tap Tapered WP	
Guided Screw Tap 6.0/WP Ø 6 7-13 mm	
Guided Screw Tap Tapered 6.0	
Guided StartDrill/Countb former MkIII RP	
Guided Screw Tap RP Ø 4 7-13 mm	
Guided Twist Drill 3.2x(10+)7-13 mm	
Guided Twist Drill 3.4x(10+)7-13 mm	
Guided Twist Drill 3.8x(10+)7-13 mm	
Guided Twist Drill 4.2x(10+)7-13 mm	
Guided Twist Drill 5x(10+)7-13 mm	
Guided Twist Step Drill Ø 2.4/2.8 (10+) 7-13 mm	
Guided Twist Step Drill Ø 2.8/3.2 (10+) 7-13 mm	
Guided Twist Step Drill Ø 3.2/3.6 (10+) 7-13 mm	
Guided Twist Step Drill Ø 3.8/4.2 (10+) 7-13 mm	
Guided Twist Step Drill Ø 4.2/4.6 (10+) 7-13 mm	

Nome prodotto	Connessioni al manipolo
Guided Twist Step Drill Ø 2.4/2.8 (10+) 7-18 mm	
Guided Twist Step Drill Ø 2.8/3.2 (10+) 7-18 mm	
Guided Twist Step Drill Ø 3.2/3.6 (10+) 7-18 mm	
Guided Twist Step Drill Ø 3.8/4.2 (10+) 7-18 mm	
Guided Screw Tap NobelActive® NP 8.5-10 mm	
Guided Screw Tap NobelActive® RP 4.3 8.5-10 mm	
Guided Screw Tap NobelActive® RP 5.0 8.5-10 mm	
Guided Twist Drill Tapered 2 x (10+) 8-16 mm	
Guided Start Drill	
Guided Counterbore NobelReplace® 3.5	
Guided Counterbore NobelReplace® 4.3	
Guided Counterbore NobelReplace® 5.0	
Guided Counterbore NobelReplace® 6.0	
Guided Screw Tap Tapered NP	
Guided Pilot Twist Drill 1.5x(10+)10-15 mm	
Guided Twist Step Drill 4.2/5.0 (10+) 7-13 mm	
Guided Twist Step Drill 4.2/5.0 (10+) 7-18 mm	
Guided Screw Tap NobelActive® WP 5.5 7-8.5 mm	
Guided Screw Tap NobelActive® WP 5.5 10-15 mm	
Guided Twist Step Drill Ø 4.2/4.6x(10+)7-18 mm	
Guided Screw Tap NobelParallel™ CC 3.75 7-10 mm	
Guided Screw Tap NobelParallel™ CC 3.75 11.5-13 mm	
Guided Screw Tap NobelParallel™ CC 3.75 15-18 mm	
Guided Screw Tap NobelParallel™ CC 4.3 7-10 mm	
Guided Screw Tap NobelParallel™ CC 4.3 11.5-13 mm	
Guided Screw Tap NobelParallel™ CC 4.3 15-18 mm	
Guided Screw Tap NobelParallel™ CC 5.0 7-10 mm	
Guided Screw Tap NobelParallel™ CC 5.0 11.5-13 mm	
Guided Screw Tap NobelParallel™ CC 5.0 15-18 mm	
Guided Screw Tap NobelParallel™ CC 5.5 7-10 mm	
Guided Screw Tap NobelParallel™ CC 5.5 11.5-13 mm	
Guided Screw Tap NobelParallel™ CC 5.5 11.5-15 mm	
Guided Tissue Punch NP-sleeve (3.75)	
Guided Tissue Punch WP-sleeve (5.5)	
Guided Counterbore NobelParallel™ CC 3.75	
Guided Counterbore NobelParallel™ CC 4.3	
Guided Counterbore NobelParallel™ CC 5.0	
Guided Counterbore NobelParallel™ CC 5.5	
Guided Dense Bone Screw Tap NobelActive® NP 8.5-18 mm	
Guided Dense Bone Screw Tap NobelActive® RP 4.3 8.5-18 mm	
Guided Dense Bone Screw Tap NobelActive® RP 5.0 8.5-18 mm	
Guided Dense Bone Screw Tap NobelActive® WP 5.5 7-8.5 mm	
Guided Dense Bone Screw Tap NobelActive® WP 5.5 10-15 mm	
Guided Drill Tapered NP	
Guided Drill Tapered RP	
Guided Drill Tapered WP	
Guided Drill Tapered 6.0	

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Compatibilità NobelProcera®/Procera® Abutment Zirconia

Abutment in zirconio - con Unigrip™					
Torque	Torque	Torque	Torque	Torque	Torque
Connessione conica	NP	Clinical Screw CC NP	Lab Screw CC NP	Unigrip™	35 Ncm
	RP	Clinical Screw CC RP/WP	Lab Screw CC RP/WP		
Trilobata	NP	Screw Ceramic Abutment NobRpl NP	Lab Screw Implant Level NobRpl NP		
	RP	Screw Ceramic Abutment NobRpl RP/WP/6.0	Lab Screw Implant Level NobRpl RP/WP/6.0		
	WP				
	6.0				
Esagono esterno	NP	Screw Ceramic Abutment Bmk Syst NP	Lab Screw CC NP		
	RP	Screw Ceramic Abutment Bmk Syst RP	Lab Screw CC RP/WP		
	WP	Screw Ceramic Abutment Bmk Syst WP	Lab Screw Implant Level Bmk Syst WP		

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IFU3018. NobelProcera® Zirconia Implant Bridge

	Tipo di connessione											
	Esagono esterno			Interna trilobata			Connessione conica (CC)			MUA		
NobelProcera Implant Bridge in zirconia	NP	RP	WP	NP	RP	WP	6.0	NP	RP	WP	NP/RP	WP
Adattatore clinico	N/D							Metal Adapter Implant Bridge Clinical CC			N/D	
Adattatore di laboratorio	N/D							Metal Adapter Implant Bridge Lab CC			N/D	
Vite clinica	Screw Ceramic Abutment Brånemark System®							Omnigrip™ Clinical Screw CC			Prosthetic Screw Multi-unit Abutment Omnigrip™ Mini	
Vite da laboratorio	Lab Screw Implant Level External Hex			N/D (*)				Omnigrip™ Lab Screw CC Lab Screw Implant Level Conical Connection			Lab Screw Multi-unit Abutment Omnigrip™ Mini	
Replica dell'impianto/MUA	Implant Replica Brånemark System®			Implant Replica NobelReplace®				Implant Replica Conical Connection			Multi-unit Abutment Plus Replica	Multi-unit Abutment Plus Replica
	Elos Accurate Analog for Printed Model External Hex			Elos Accurate Analog for Printed Model Tri-Channel				IOS Model Implant Replica CC			Elos Accurate Analog for Printed Model Multi-unit	Abutment Replica Multi-unit Bmk Syst Elos Accurate Analog for Printed Model Multi-unit
Analoghi di protezione	Protection Analog Bmk Syst			Protection Analog NobelReplace®				Protection Analog/Drill Guide CC			Protection Analog Multi-unit	Protection Analog Multi-unit Protection Analog M-u Bmk Syst
MUA	Multi-unit Abutment Bmk Syst 17° Multi-unit Abutment Bmk Syst 30° Multi-unit Abutment Bmk Syst Zygoma Abutment Brånemark System® Zygoma Zygoma 17° Brånemark System® Zygoma 17° 45° Multi-unit Abutment Ext Hex 60° Multi-unit Abutment Ext Hex			Multi-unit Abutment NobRpl 17° Multi-unit Abutment NobRpl 30° Multi-unit Abutment NobRpl				Multi-unit Abutment Plus Conical Connection 17° Multi-unit Abutment Plus Conical Connection 30° Multi-unit Abutment Plus Conical Connection Multi-unit Abutment Xeal™ Conical Connection 17° Multi-unit Abutment Xeal™ Conical Connection 30° Multi-unit Abutment Xeal™ Conical Connection			N/D	
Impianti	NobelSpeedy® Groovy	Brånemark System® Mk III	NobelSpeedy® Groovy	Replace Select™ TC				Impianti Nobel Biocare dotati di connessione conica (CC).	Impianti Nobel Biocare dotati di connessione conica (CC).	Impianti Nobel Biocare dotati di connessione conica (CC).	N/D	
	NobelSpeedy® Shorty	Brånemark System® Mk III TiUnite® NobelSpeedy® Groovy NobelSpeedy® Shorty Impianto Zygoma NobelZygoma™	NobelSpeedy® Shorty									

IFU3020. NobelProcera® Zirconia N1™ Base

Compatibilità NobelProcera® Zirconia NB N1™ Base – Vite protesica – Nobel Biocare N1™ Base Xeal TCC Tri

Tipo di protesi	Vite protesica	Cacciavite	Nobel Biocare N1™ Base Xeal™ TCC Tri
Nobel Procera® Zirconia Abutment/ Implant Crown NB N1™ Base NP	Prosthetic Screw NobelProcera® Zirconia Nobel Biocare N1™ Base NP	Omnigrip™ Mini	Nobel Biocare N1™ Base Xeal™ TCC Tri NP
Nobel Procera® Zirconia Abutment/ Implant Crown NB N1™ Base RP	Prosthetic Screw NobelProcera® Zirconia Nobel Biocare N1™ Base RP		Nobel Biocare N1™ Base Xeal™ TCC Tri RP

Compatibilità NobelProcera® Zirconia NB N1™ Base – Vite da laboratorio - Base Replica

Tipo di protesi	Vite da laboratorio	Base Replica Nobel Biocare N1™ Base	IOS Base Replica Nobel Biocare N1™ Base
Nobel Procera® Zirconia Abutment/ Implant Crown NB N1™ Base NP	Lab Screw NobelProcera® Zirconia Nobel Biocare N1™ Base NP	Base Replica Nobel Biocare N1™ Base Tri NP	IOS Base Replica Nobel Biocare N1™ Base Tri NP
Nobel Procera® Zirconia Abutment/ Implant Crown NB N1™ Base RP	Lab Screw NobelProcera® Zirconia Nobel Biocare N1™ Base RP	Base Replica Nobel Biocare N1™ Base Tri RP	IOS Base Replica Nobel Biocare N1™ Base Tri RP